MD

# LUXIFLEX SENSOR

**MIMSAL**

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**LUXIFLEX SENSOR**  
**LUXIFLEX SENSOR PLUS**

*cod. I0104AOLEDSR*  
*cod. I0104NLSR*

The design of the light fixture is highly versatile. It is suitable for medical examination and has many features that makes it ideal for use in medical centres and veterinary clinics.

The optimum working distance between the light bulb and the object being lit up is 50 cm.

### 1. Installation

- Requires a MIMSAL mount, which is installed before the lamp is placed in position.
- A table clamp mount is supplied and wall or ceiling mounts or a rotating base can be supplied as optional accessories.
- MIMSAL cannot guarantee the life and functionality of the product where third party accessories are used.
- ON/OFF via Proximity Sensor. Move your hand about 8 cm over the sensor.

### 2. Caution

- Always disconnect the lamp from the power supply when cleaning or carrying out maintenance work.
- Clean either dry or using a weak solution of normal soap and water. Avoid using alcohol, acids or any type of organic solvent.
- Do not leave on for longer than 8 hours at a time.

### 3. Troubleshooting and maintenance

- Only for qualified electrical engineers or other qualified personnel.

Notify by the user / patient to MIMSAL and the competent authority of the country, any serious incident that occurs in relation to the use of the equipment

MIMSAL only accepts responsibility for maintenance, repair or changes carried out by MIMSAL staff using original MIMSAL parts.

The manufacturer reserves the right to change the characteristics of its products without prior notice.

	LUXIFLEX SENSOR	LUXIFLEX SENSOR PLUS
Light bulb	LED 6W 15°	LED 10W 14°
Number of LEDs	3	4
Useful life	50.000 h	50.000 h
Lux at 50 cm	15.000 lux	35.000 lux
Colour temperature	4.500° K	4.500° K
Colour Rendering Index	80%	97%
Electrical Protection Class	Class II	Class II
Total Length	85 cm	85 cm
Color	White	White
Power	110/230V	110/230V
Frequency	50/60 Hz	50/60 Hz

**LUXIFLEX SENSOR**  
**LUXIFLEX SENSOR PLUS**
*cod. I0104AOLEDSR*  
*cod. I0104NLSR*

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