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Dear Customer,

Congratulations on having chosen a quality RUBENS bedside cabinet which will certainly fully meet your expectations. We would like to take this opportunity to thank you for showing such confidence in our company and purchasing one of our products.

This instruction manual provides important information regarding all functions of RUBENS bedside cabinets and also describes their: assembling, care, maintenance and operating the adjustment mechanism.

Read these instructions carefully and follow them exactly in order to provide reliable service and avoid damage or incorrect operation.

As manufacturer of RUBENS bedside cabinets we reserve the right to change their technical specification, without obligation to update or replace those products.

If you have any questions about RUBENS bedside cabinets, please do not hesitate to contact your local distributor or *Elbur sp. z o.o. sp.k.* customer service through our website.

We will do our best to fulfil your needs.

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1.0. Product description

1.1. Purpose and range of application

RUBENS bedside cabinets are great addition to ELBUR nursing beds type PB offered by our company *Elbur sp. z o.o. sp.k.* They are intended to alleviate and compensate injuries and disabilities for older, frail and handicapped people. They allow patients to store their private things on them and simplify eating meals, reading books, newspapers or using a computer. They are easy to clean and high quality workmanship.

RUBENS bedside cabinets are made of furniture board finished with PVC veneer. They can be located on both sides of the nursing bed, due to its construction, without loss of functionality - doors and drawer can be opened from both sides, what is very useful. Those products stand on four castors – two of them can be locked. Private things patient can store on the open shelf or inside the closed space, which is split on two levels. Only difference between RUBENS 1 and RUBENS 2 is that patient or user can swivel and adjust height of table tray in RUBENS 2.

Our products meet the varying needs of patients and professional caregivers. In order to meet customer expectations we care about attractive design, as well as matching furniture systems in every place of use. That is why we offer our customers wide choice of decors, although our standard is dark beech décor.

RUBENS bedside cabinets described in this document compliant with the requirement of the EEC Directive on medical devices 93/42/EEC dated 14 June 1993 with subsequent amendments.

1.2. Technical specification

Parameters	Bedside cabinet model	
	RUBENS 1	RUBENS 2
Height [cm]	86,0	86,0
Depth [cm]	45,0	45,0
Width [cm]	53,0	53,0
Width with bedside table tray [cm]	118,5	123,0
Height adjustment range of the bedside table tray [cm]	---	71,0 – 99,0
Castors diameter [cm]	Ø5	Ø5
Safe working load of the bedside table tray [kg]	6,0	6,0
Weight [kg]	32,4	38,6

1.3. Labelling of RUBENS bedside cabinets

1.3.1. Name plates

Name plate data are defined in the following order:

- model name,
- date of manufacture,
- safe working load of the bedside table tray,
- manufacturer,
- serial number,
- information and warning symbols.

1.3.2. Explanation of symbols used on name plates

- do not use outdoors
- consult the instruction for use for warnings and precautions
- follow the instruction for use
- EC Declaration of Conformity
- manufacturer
- safe working load

2.0. Safety information

2.1. Marking and safety signs

Warning

Indications of potential risks.

Note

Information and tips regarding the safe use.

2.2. Potential risks and warning messages

Warnings

-  *Use the bedside cabinet in accordance to normal operating procedure and its purpose.*
-  *Before start using the bedside cabinet, read instruction carefully and familiarize with rated values provided at name plates, labels and in this manual. It helps to prevent any damage and risk occurring due to incorrect operation.*
-  *Keep the instruction manual for future reference and have access to it at any time – it provides information for operator and patient regarding the convenient handling and safe operation.*
-  *Safe working load of the bedside table tray is 6 kg and it cannot be exceeded. Do not lean on the table – it can cause damage of adjustment mechanism and lead to serious injuries!*
-  *Before every use make sure that the bedside cabinet is in proper and faultless condition.*
-  *Only use technically perfect bedside cabinet - contact manufacturer or distributor if the exchange or repair of faulty parts have to be made.*
-  *If damage or malfunction is suspected, take the bedside cabinet out of service immediately and label it as non-conforming product.*
-  *Use only original Elbur sp. z o.o. sp.k. accessories – modification of bedside cabinets is strictly forbidden without authorization of our company.*
-  *Make sure that other attached equipment can work in a secure and safe manner (do not hesitate to contact manufacturer with questions).*
-  *Make sure that castors have their brakes applied when location of bedside cabinet is not going to change.*
-  *Close tray down before relocating the bedside cabinet. Do not load bedside table tray during pushing it closer to the nursing bed.*
-  *Before adjusting the height of bedside table tray, make sure there are not any obstacles, such as furniture, window sill etc.*
-  *Make sure that no cable from electric devices is rolled over during relocating bedside cabinet.*
-  *Make sure that no cable from electric devices is damaged, stretched or crushed between bedside cabinet parts.*
-  *Do not clean bedside cabinets with a water jet or a high-pressure cleaner. Clean only with a moist towel.*
-  *Never use scouring agents, abrasive cleaning agents, stainless steel cleaners or scouring pads, because they can damage the surface.*
-  *Remove quickly stains, liquid from shelf or top of the bedside cabinet – otherwise it can cause dampness of particle board.*
-  *Read instruction carefully before commencing assembly.*

- ⚠ *Verify the shipment contents with your order before commencing assembly to make sure that you have got all necessary parts and components.*
- ⚠ *Remember to use proper tools.*
- ⚠ *It is obligated with every maintenance to perform regular inspections, to ensure the safe condition of the bedside cabinet.*

The safety information indicated above and found in this instruction manual must always be strictly observed.

3.0. Scope of delivery

Before commencing assembly of the bedside cabinet, please read instruction carefully and verify the shipment contents with your order. Please contact your distributor or Elbur sp. z o.o. sp.k. customer service, if you miss any component or get something in bad condition

If you place an order for RUBENS 1 or RUBENS 2 you should expect to receive:

- bedside cabinet with door both sides, drawer (inserts removable) and bedside table tray,
- 4 handles (2 drawer, 2 for doors),
- 1 handle for bedside table tray,
- 4 castors (2 of them can be locked),
- 1 wooden shelf,
- 16 screws to assembly castors,
- 8 long screws to assembly handles,
- 2 short screws to assembly handle for bedside table tray.

4.0. Accessories

Warning

- ⚠ *Use only original Elbur sp. z o.o. sp.k. accessories – modification of bedside cabinets is strictly forbidden without authorization of our company.*

There is possibility to order plastic cover for top of the bedside cabinet, which helps protecting it from liquid spill.

5.0. Spare parts

Spare parts can be ordered from distributor or Elbur sp. z o.o. sp.k. For reasons of safety, it is allowed to use only original spare parts or accessories from our company. Otherwise, we will assume no liability for any accidents, damage, injury and risks that come about through the use of other spare parts or accessories.

6.0. Assembly

Before commencing assembly of the nursing bed, please read carefully instruction manual and verify the shipment contents with your order (chapter 3.0). Please contact your distributor or *Elbur sp. z o.o. sp.k.* customer service, if you miss any items or get something in bad condition.

Warnings


 *In order to avoid wrong assembly, read carefully instruction manual and pay attention to labelling on the bedside cabinet. Remember also to use proper tools.*

6.1. Location requirements

- there must be enough space available for adjusting the bedside cabinet tray, its position and height,
- RUBENS bedside cabinets can be located on both sides of the nursing bed, due to its construction, without loss of functionality.

6.2. Castors assembly

- carefully lay the bedside cabinet down on its side (where is not bedside table tray),
- fix castors from the bottom – use 4 screws for each castor,
- two castors have locking functions – they should be fixed diagonally,
- after fixing all castors, bedside cabinet can stand on them, two castors should be locked now.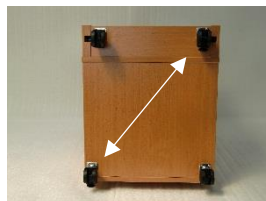


Note

 *There should be access to castors at all the time. They have to be placed along the bedside cabinet before locking them.*

For the purpose of activating (1) the brake push the foot lever down as far as it goes.

For the purpose of deactivating (2) the brake push the foot lever toward as far as it goes.

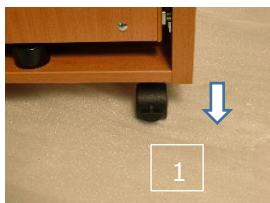6.3. Handles assembly

- bring drawer out a bit, attach handle to it and screw two long screws from inside – do the same on the other side of drawer
- open door, attach handle to it and screw two long screws from inside – do the same for the second door,
- attach handle to bedside table tray and screw to short screws two assembly it.

It is recommended to assembly handles for drawer and doors on both sides of the RUBENS bedside cabinet, because it allows using it in case of change its position in regard to the nursing bed location.

7.0. Using RUBENS bedside cabinets

7.1. Open tray out - RUBENS 1

Firstly, push handle down (3) and pull it right up (4). It releases locking mechanism – then bedside table tray can be opened outwards and placed parallel to the floor (5).

7.2. Adjustment of bedside table tray level - RUBENS 1

Please adjust black knobs (6) to adjust level of bedside table tray. It has to be done on both ends at the same time and each knob has to be at the same level.


7.3. Close tray down - RUBENS 1

Firstly, pull tray right up using handle and slowly lower it. Then push handle down until it stops (7) and push it forward (8) to the side of bedside cabinet to lock it in the limited space (normal position).

7.4. Open tray out - RUBENS 2

Firstly, push handle down (9) and pull it right up (10). It releases locking mechanism – then bedside table tray can be opened outwards and placed parallel to the floor (11).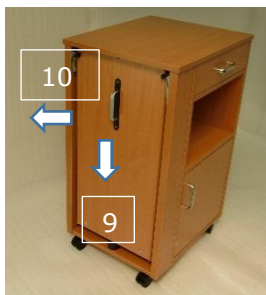


7.5. Height adjustment and swiveling of the bedside table tray - RUBENS 2

Warning

- ⚠ Before adjusting the height of bedside table tray, make sure there are not any obstacles, such as furniture, window sill etc.

The biggest advantage of the RUBENS 2 is that there is possibility to adjust height and angle level of bedside table tray.

Pull two black levers (12) to release gas spring. It allows to adjust height in demanded range (release levers when the height is good enough for patient). The adjustment range is between 66 and 94 cm (13).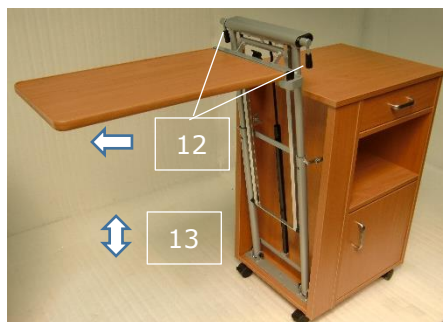


Press button (14) and swivel bedside table tray – it clicks into required position after releasing that button.

There are three positions available to achieve for bedside table tray.

Warnings

- ⚠ Do not load bedside table tray before locking it in the proper position and during pushing it closer to the nursing bed.
- ⚠ Do not put any drinks or meals on bedside table tray when it is swiveled.
- ⚠ Safe working load of the bedside table tray is 6 kg and it cannot be exceeded. Do not lean on the table – it can cause damage of adjustment mechanism and lead to serious injuries!
- ⚠ Close tray down before relocating the bedside cabinet.

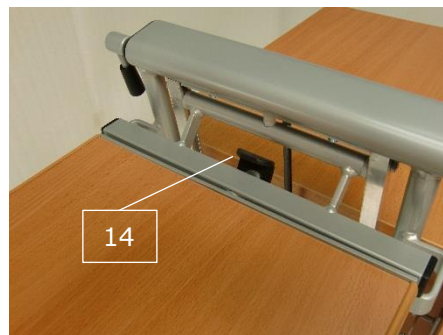
7.6. Close tray down - RUBENS 2

Firstly, pull tray right up using handle and slowly lower it. Then push handle down until it stops (15) and push it forward (16) to the side of bedside cabinet to lock it in the limited space (normal position).

7.7. Insert / pull out the drawer

Customer gets the bedside cabinet with the inserted drawer. To pull it out completely, please twist the wooden locking component and hide it in the free space under the top of bedside cabinet. To insert drawer back, please twist the wooden locking component and hide it in the free space under the top of bedside cabinet, then move it forward and make sure it cannot be pulled out.

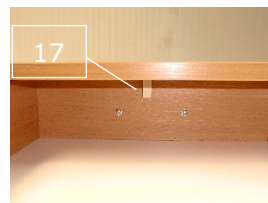7.8. Middle shelf location

It all depends on which side of the nursing bed is bedside cabinet located. Middle shelf (18) allows to split closed space for two parts and still leave enough space for bigger items, bottles for example. It can be easily moved on wooden guide – it is not fixed permanently.

8.0. Maintenance

Warnings

- ⚠ *Do not clean bedside cabinets with a water jet or a high-pressure cleaner. Clean only with a moist towel.*
- ⚠ *Never use scouring agents, abrasive cleaning agents, stainless steel cleaners or scouring pads, because they can damage the surface.*
- ⚠ *Remove quickly stains, liquid from shelf or top of the bedside cabinet – otherwise it can cause dampness of particle board.*

8.1. Disinfecting

All mild, not aggressive disinfectants set out in EN 12720+A1 can be wiped on the bedside cabinets to clean them. They must be cleaned and washed down with disinfectant in case of a patient change or in other situation required it.

8.2. Cleaning and care

All components of the bedside cabinet are made from the best quality material. The surface of steel tubes is coated with a durable polyester powder layer. All wooden or wood-based parts are protected with non-hazardous varnish. The bedside cabinet components can be easily cleaned and maintained according to the effective hygienic rules in specific areas by disinfecting wiping. Adhering to the described below rules of the furniture maintenance ensures its long life and keeps its nice appearance.

CLEANING AND CARE		
NO.	Components	Cleaning method
1.	Steel tubes and coated metal parts	Use a slightly moisturized cloth and normal, mild, not aggressive agents for cleaning furniture
2.	Wooden and wood-based elements	
3.	Plastic components	

9.0. Service

Warnings

- ⚠ *If damage or malfunction is suspected, take the bedside cabinet out of service immediately, label and protect it properly as a non-conforming product.*
- ⚠ *Contact manufacturer or distributor if the exchange or repair of faulty parts have to be made.*

It is obligated to perform regular inspections, to ensure the safe condition of the bedside cabinet:

- First inspection – before first use;
- Service inspection – service should be performed after every repair.

9.1. Troubleshooting

DEFECT	POSSIBLE CAUSES	SOLUTION
Locking castors are not active	Castors unclean	Clean castors, remove all obstacles
	Castors damaged	Notify customer service
There is no possibility to adjust bedside cabinet tray	Gas spring damaged	Notify customer service

10.0. Transport and storage

All ordered bedside cabinets are packed on the transport palette, which can be relocated by the hand pallet truck or the forklift. Reliable package protects bedside cabinets from external damages and ensures that patient or user get a bedside cabinet in the perfect condition.

Technical specifications (*subclause 1.2.*) contain guidance of operating and storage conditions for Elbur sp. z o.o. sp.k. furniture.

Bedside cabinets shall be protected from destruction under external influences before re-storage.

Disassembly shall be performed in the opposite directions to steps specified in paragraph 6 – Assembly.

11.0. Disposal

Bedside cabinets are made of metal, wooden and plastic components. If they must be scrapped, the plastic and metal parts shall be recycled according to legal regulations. If in doubt, please contact your nearest disposal facility, local community, distributor or Elbur sp. z o.o. sp.k. customer service.

12.0. Warranty

The manufacturer gives a guarantee of 24 months for a perfect functioning bedside cabinet.

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