## Commode and shower chair KAKADU

Art. 302015, 302018, 302019 & 302020

**Gebrauchsanleitung**  
**Brugervejledning**  
**User Manual**  
**Manuale d'uso**  
**Manuel d'utilisation**

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. These user instructions contain a detailed product description and important guidelines for correct use of the product. You must read the instructions carefully before using the product. Please always observe all guidelines, in particular the security instructions, in order to use the product safely for a long time.

### Intended use of the device

The Kakadu is a medical device intended to be used only as seating support by individuals who require such help during bathing, showering and defecating. It is also a transport option intended to move patients from and to bathrooms and toilets. The device serves a fundamental function and makes the life of both the physically challenged user and the assistant much easier. The device should only be used indoors. Any other use is prohibited.

### Indications

Commode wheelchairs are indicated for people with mobility limitations who have problems with walking, standing, or sitting without stable support. The devices are dedicated to persons who, because of their disabilities, can't perform their daily hygienic activities in regular shower cabins, bathtubs and who are unable to use regular toilets, even after modifications.

The commode is indicated to move patient to and from bathroom in a sitting position independently, or with the help of an attendant and to enable hygienic activities such as body washing or defecation. Using the product for purposes other than those indicated above, is prohibited.

### Contraindications

The commode wheelchair should not be used by people who cannot maintain a stable sitting position. Contraindications cannot be related to one specific clinical picture, and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist. In addition, observe the safety information in Chapter 2.

If you have any questions, please contact your dealer or Mobilex A/S

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Phone: +45 87 93 22 20  
Telefax: +45 87 93 17 77  
Email: info@mobilex.dk

## 2. Security instructions

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During the assembly or during the different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device.
- The device must only be used on a stable surface.
- All wheels MUST be in contact with the floor at ALL TIMES during use. This will ensure that the device is properly balanced and should avoid accidents.
- Do not stand on footrests, and do not sit on the edge of the seat with the legs on the footrests
- When using the device in a stationary position, the brakes MUST be locked.
- Each device must not be loaded with more than specified in the technical data sheet.
- The device is only to be used indoors and on flat surfaces without any obstacles.
- Always maintain a stable sitting position when using this device and users should not be left on the device unattended. Only remove the hole cover when chair is used for defecating.
- When the toilet cut-out is mounted on the seat, the bucket with lid must also be mounted.

## 3. Description

### Product description

The device is supplied in a carton, detached into few elements for easier transportation and assembly. The contents are:

- ✓ 1 main frame with seat and 1 detachable back frame with armrests
- ✓ 2 detachable Footrests
- ✓ 4 detachable wheel assemblies consisting of wheel, brake and vertical assembly tube
- ✓ 4 bolts and 4 nuts for wheel assembly
- ✓ 1 defecation bucket and 1 PU cover for the middle hole of the seat
- ✓ This user manual

## Function and assembly description

The Kakadu toilet and bath chair is installed as shown in the picture opposite. We recommend the following order:

- Take all parts out of the box and check that all parts are there. If anything is missing, contact the dealer immediately before further assembly.
- Turn the frame upside down and place it on the floor.
- Insert the wheel mounting tubes into each tube. 2 holes are drilled (A & B) in both the mounting tube on the wheel and in the frame. The wheels can easily be mounted at 3 different heights. Make sure that all wheels are mounted at the same height. Tighten the bolt securely.
- Now turn the frame over. Lock the brakes on the wheels before continuing.
- Fit the spine tubes in the 2 mounting holes at the back of the frame. Secure the back by turning the handles.
- Fit the lid to the bucket. Slide it into the rails fitted under the seat. This is done from the front of the chair. Then close the cut-out with the removable PU lid.
- Fit the footrests by inserting the plastic plug at the top of the footrests into the tube ends in the frame front. Then turn the footrest until it locks.

## Use of the brakes

- 1 Always brake when you don't want to move the chair.
- 2 Never leave the Kakadu shower chair and toilet wheelchair unbraked when a person is sitting on it.
- 3 Never leave the Kakadu shower chair and toilet wheelchair unbraked on slopes.
- 4 To lock the brakes, press the brake lever on the wheels with your foot until it engages (see picture on the right).
- 5 To release the brakes, lift the brake levers on the wheels with your foot (see picture on the right).

*Lock the brake*

*Release the brake*

### Folding up the armrests

- This feature can help make it easier for the user to get in and out of the Kakadu toilet and bath chair.
- Raising the armrests makes it easier to perform personal hygiene while showering.
- Never leave the Kakadu toilet and slanted armchair unattended when a person is using it.
- To raise the armrest, tilt the armrest backwards by lifting it at the front.
- To return the armrest to its original position, simply tilt it forward again until it is firmly horizontal for use.

### Removing and fitting the footplates

- This feature, together with the high foldable footplates, can make it easier for the user to get in and out of the shower stall wheelchair. It also allows for easier personal hygiene while showering.
- Do not move the wheelchair with the footrests removed when a person is using it. When moving the chair, always place the user's feet on the footrests.
- To release and remove the footplates, press down on the lock handle with your thumb and turn the footplates outwards.
- To release and remove the footplates, press down on the lock handle with your thumb and turn the footplates outwards (image right).
- Grasp the dismantled footrest tube by the top tube and pull it upwards out of the front frame (image right).
- To refit and secure the footrests, insert them into the hole on the front frame and turn them inwards until the locking lever engages again.

Footrest lock handle

Footrest tube

### Adjusting the footrests to the desired height

- The height of the toilet and bath seat footrests must be adapted to the user.
- The height is correctly adjusted when the user sits on the wheelchair, the thighs rest correctly on the seat and both feet rest on the footrests.
- To adjust the height of the footrests, remove them from the toilet and bath seat.
- Adjust the height of the footrests by opening the flap (picture) on the footrests and then moving the inner tube of the outer tube up or down.
- Close the flap on the footrests again.

### Adjusting the desired seat height


- The seat height must be adapted to the user. The chair is correctly adjusted when the user sits on the chair with both feet on the floor.
- Adjust the seat height only when no one is sitting in the chair.
- To adjust the seat height, remove screw A. Now the telescopic tube can be moved with the 5" wheel in the frame and can be adjusted in height (4 positions)
- When the desired height is reached, reinsert the bolt into hole A or B (depending on the height) and secure with the corresponding nut.
- Always set all 4 telescopic tubes at the same height!
- If the chair is equipped with 24" rear wheels (optional), the height of the axles of the 24" wheels must be adjusted so that the seat is flat and horizontal.

## 24" wheel set 302017 und 302023 (Accessories)

It is possible to easily replace the 2 rear 5" wheels with 24" wheels. All parts, including mounting brackets and brakes, are included in the accessory kit.

### Mounting of 24" wheels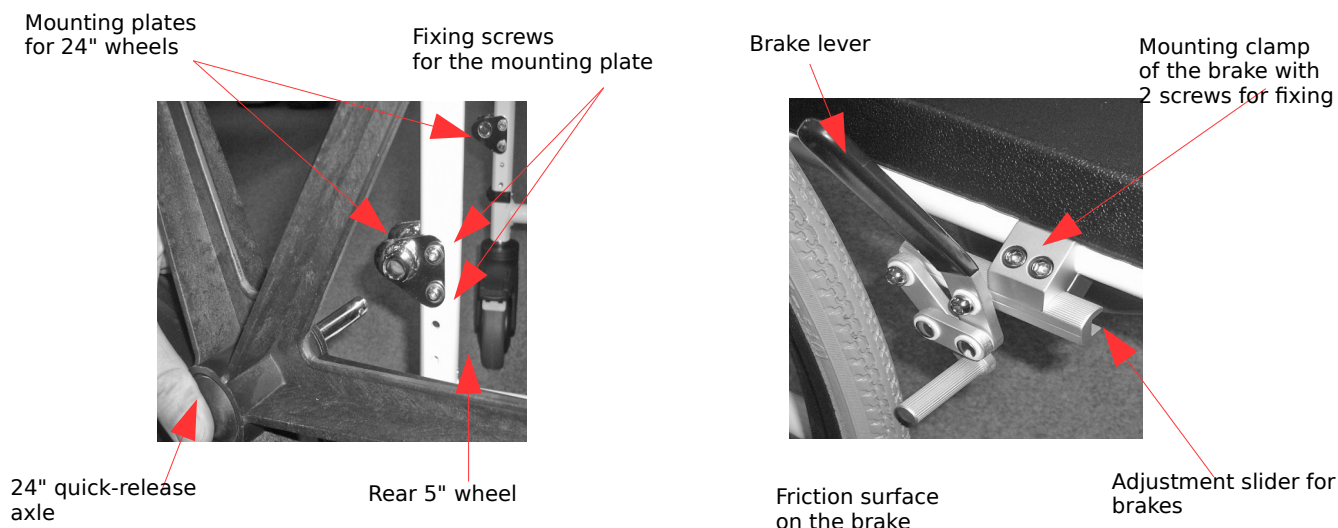

- Remove the 2 rear 5" wheels by loosening the bolts.
- Place the mounting brackets as shown in the picture above.
- Several holes have been drilled in the frame to allow a single height adjustment. Choose a height and insert and secure the mounting bolts with the corresponding washers.
- Press the quick release axle located in the middle of the wheels. Insert the axle into the brackets as far as it will go and release the push button.
- Take hold of the wheels and check that they are properly fastened.
- To remove the wheels, press the push button while pulling the wheel with the other hand.
- After fitting 24" wheels, it may be necessary to adjust the front wheels so that the chair appears horizontal.

### Brake assembly with 24" wheels:

The 24" wheel set (302017 or 302023) comes with 2 parking brakes with brackets. The brakes are mounted on the wheelchair frame and are used to lock the two rear wheels.

Attach the brake to the frame

Brake released

Brake locked


- When 24" wheels are fitted it is necessary to fit normal wheelchair brakes.
- The brakes consist of mounting brackets, the brake itself and 2 fixing bolts.
- Insert the brake adjustment profile into the smallest opening of the brake caliper.
- Place the mounting bracket on the horizontal tube of the chair as shown in the picture.
- Move the brake until the friction pin almost touches the tyre.
- Secure the bolts so the brake is stable.
- To apply the brakes, push the brake lever forward. To release the brake, pull the lever backwards.

To secure your Kakadu bath and toilet chair against accidental rolling away: to lock the wheelchair, push the brake lever on the right and left wheels forward as far as possible. To release the brakes, pull both brake handles back again.


**Warning! Always use the brakes on both sides of the wheelchair for the shower-toilet. The parking brakes are not intended as service brakes and are only used to lock the Cockatoo shower and toilet wheelchair (parking position). They must never be used to brake while driving!**

### Using the toilet bucket

- The toilet bowl is designed for people who have difficulty using regular toilets.
- To use the toilet bowl, remove the seat plate (see picture).
- Remove the lid of the toilet bowl.
- After use, put the lid back on the bucket and remove it from the wheelchair (see picture).
- Empty and clean the bucket and put it back in the wheelchair (see picture).
- Never leave the bucket full or uncleaned.
- Cover the toilet bowl again with the seat plate.
- When the toilet cut-out is mounted on the seat, the bucket with lid must also be mounted.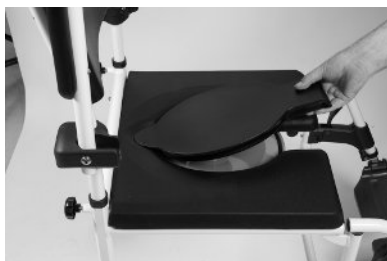

*Seat plate removal*

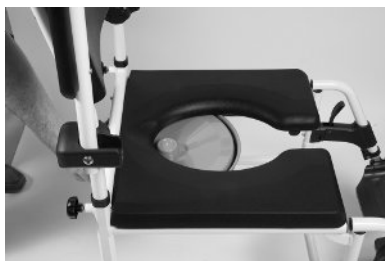*The bucket is removed from the rear*

*Empty and clean the bucket*

### Fastening of seat belt or safety bar (optional).

A safety belt (art. no. 233012) or a safety bar (art. no. 302126/302127/302149) can be fitted very easily. For 302018-20) can be fitted very easily.

The safety bar is fitted to the front of the armrest by removing the two black caps and the two front screws on the armrest and inserting the two brackets for the safety bar, which must then be secured with the screws removed.

The bracket itself can be secured in place with a click bracket. To fit the safety belt, make sure the belt has loops at the ends of the belt. These loops can be easily attached to the wheelchair frame by removing the backrest from the wheelchair and placing the loops on the wheelchair. the ends of the loops around the frame (image right).

Make sure the strap is fastened with the closing mechanism facing forward and is not twisted. Then fit the backrest back onto the shower-toilet wheelchair and screw it in place. Done!

The safety belt is fitted (image right).

*Open the backrest*

*seat belt fitted*

### Transport recommendations (how to lift and carry)

The Kakadu commode and shower chair is not a device designed for frequent transportation. If it is necessary to transport the chair, we recommend to disassemble it in the way opposite to the assembly description above. Chair should be packed in the way that it is stable and there is no risk of sudden and dangerous movements of the transported device.

If you need to carry the assembled shower chair, then we recommend to detach the footrests and lift the shower chair on both sides of the frame on the points shown below.


*Lifting points for the Kakadu commode and shower chair*

### 4. Technical data

Kakadu	Small 302015	Standard 302018	Large 302019	XL 302020
Seat width	38 cm	45 cm	50 cm	55 cm
Seat depth	38 cm	43 cm	43 cm	43 cm
Seat height adjust.	50 - 55 cm	50 - 55 cm	50 - 55 cm	50 - 55 cm
Seat opening	17 x 33 cm	21,3 x 40,5	21,3 x 40,5	21,3 x 40,5
Pass over height	42 - 47 cm	42 - 47 cm	42 - 47 cm	40 - 45 cm
Pass over width	32 cm	40 cm	45 cm	50 cm
Total width	49 cm	56 cm	61 cm	66 cm
Total depth	85 cm	93 cm	93 cm	93 cm
Total height adjust.	96-101 cm	96-101 cm	96-101 cm	96-101 cm
Max. Load	130 Kg	150 Kg	150 Kg	200 kg
Weight	11,00 Kg	12,90 Kg	13,50 Kg	14,10 kg

### Product label

The product label is located at the bottom of the frame

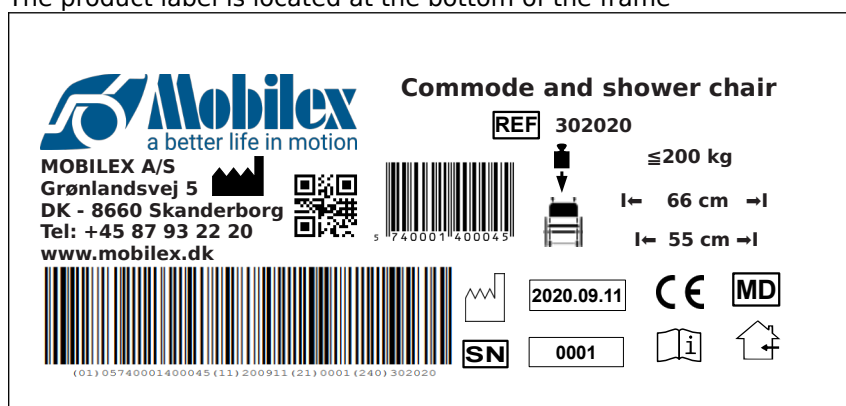Example of the product label for product no. 302020

	QR-code link to homepage		Manufacturing date
	Manufacturer		Loading force
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	User weight		Medical Device
	Only to be used indoor		

Overview icon meaning

### 5. Maintenance and care

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General retightening of the product

### **Cleaning of bucket**

Wipe with a mild soap and warm water solution, coarse soiling can be removed with mild cleaning agents free of scouring agents. Do not use water with a temperature more than 80°C.

When a steam cleaner is used. Removed iron hook from the bucket, clean the bucket after laying it flat, please do not exceed 10 minutes. Please do not place the bucket at an angle to avoid deformation during cleaning.

### **Cleaning the wheels**

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

### **Warning**

 If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated with lubricating oil.

### **Warning**

 When used in highly chlorinated environments extra strong corrosion may occur and the warranty will be void.

### **Warning**

 Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

**Recommended service life:** Normally 10 years depending on use.

### **Storage**

We recommend storage in a dry place.

### **Reuse**

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

## **6. Most common problems and solutions**

<i>Symptoms</i>	<i>Possible cause</i>	<i>What to do?</i>
The chair is not stable and can't reach horizontal position	The wheels are adjusted to different heights	Adjust all wheels to the same height
The wheels do not turn smooth and easily	The space between wheels and forks is filled up with dirt	Clean the space between wheels and forks of all unintended objects and elements

## **7. Disposal of the product**

The product can not be disposed of with household waste but must be brought to the local recycling centre.

## **8. Guarantee terms**

Mobilex A/S offers 2 years of warranty for damages A/S caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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**ENGLISH**

Declaration of Conformity

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product are classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

**Intended purpose**

Indicated for people with mobility limitations who have problems with walking, standing or sitting without stable support

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Mobilex A/S - Grønlandsvej 5 - DK-8660 Skanderborg - www.mobilex.dk
Mobilex A/S - Grønlandsvej 5 - DK-8660 Skanderborg - [www.mobilex.dk](http://www.mobilex.dk)