

INSTRUCTIONS FOR USE

Back rest

Product code: JMC-C 7000




1. Characteristics

Back rest is made of powder coated steel frame. It consists of a base part with ratchet system and a backrest part which is made of air-permeable linen material with extra pillow which increases lying comfort. Back rest can be adjusted to individual patient's needs while lying or sitting owing to ratchet system.

Elements of the back rest:

- a. ratchet system
- b. backrest
- c. pillow

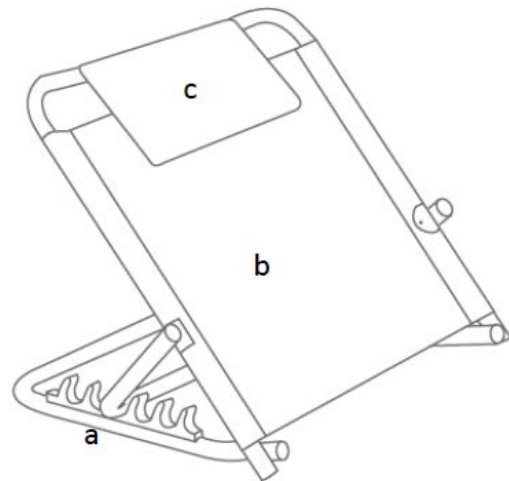

2. Application

Back rest is ancillary equipment designed for patients with reduced mobility and who are in lying position permanently which prevents or hinders the unassisted performance of daily living activities. Back rest helps to maintain half-sitting position without having to be hold by the other person. It is also used in order to tilt the patient to erect position by gradually accustoming the cardiovascular system to sitting position. The back rest is recommended for use at home and in hospitals.

3. Adjustment

Inclination angle adjustment

Back rest can be adjusted within the range of 25-27 cm.

4. Technical data

Overall width	54.5 cm
Inner width	50 cm
Overall depth	50 cm
Inclination angle adjustment	25-57 cm
Weight	2.7 kg
Maximum capacity	100 kg
Frame colour	white

5. Notes

Timago International Group does not bear responsibility of improper use of the back rest, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

Reviews

Make visual inspection of all the joining elements (clamps, hinges) of the back rest at least once every three months including:

- condition of ratchet mechanism
- stability and look of the frame

Cleaning

Control the technical condition of the back rest regularly:

- Clean the frame using only water and mild detergents. Dry thoroughly.
- **Do not use** for cleaning: strong polishing agents and detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).










7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group
Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12
43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00
F.: +48 33 499 50 11
E.: info@timago.com



Rev. II, date 20.05.2021