

INSTRUCTIONS FOR USE

Steel wheelchair

Product code: FS 901



1. Characteristics

The wheelchair is made of powder coated steel frame reinforced with double cross bar. The wheelchair is equipped with durable, nylon-cotton seat, attendant brakes, folding backrest, profiled armrests, and steel swing away footrests with 3-stage legrest angle adjustment. The wheelchair is supplied with pneumatic rear wheels (20") with plastic handrim and solid rubber front wheels 6" in diameter.

The set includes:


- Frame with seat
- Protective strap
- Pump

In the pocket at the back of backrest:

- Footrest adapter for legrest angle adjustment – 2 pcs
- Set of keys

Elements of the wheelchair:

- a. Hand grip
- b. Attendant brake
- c. Backrest lock
- d. Armrest
- e. Wheel lock
- f. Handrim
- g. Footrest
- h. Footrest adapter

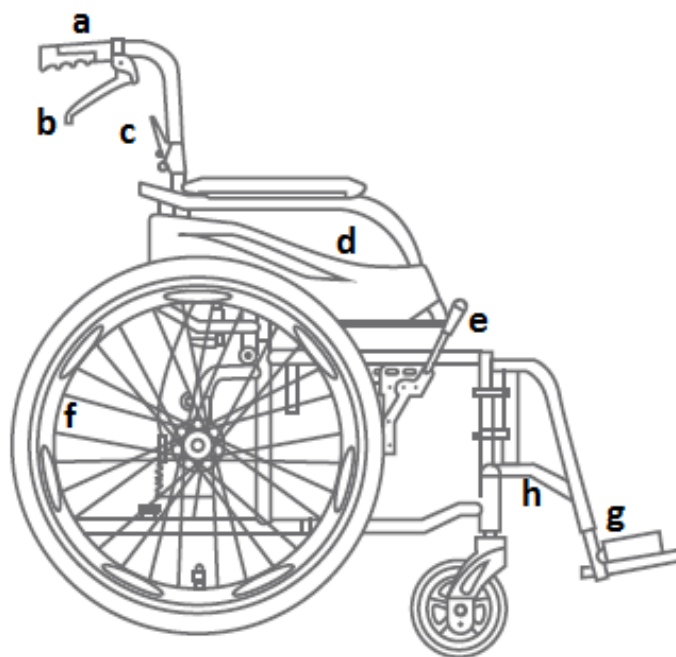

2. Application

The wheelchair is designed for people who have problems with unassisted moving and for patients with dysfunctions and reduced mobility. This self propelled wheelchair can also be pushed by the

accompanying person. The wheelchair is suitable for indoor and outdoor use.

3. Assembly

Step 1. Unfolding the wheelchair

Place the wheelchair on the ground. Firstly, push the footrests downwards. Unfold the wheelchair using the armrests. Then, press the front and rear edge of the seat so that all the elements fit together.

Step 2. Backrest

Lift the backrest upwards and lock it with backrest lock.

Step 3. Footrests

Firstly, place the footrest adapter (h) into the right and left opening under the footrest with bigger notch directed upwards (with metal pin directed upwards). The adapters can be found in the pocket at the back of the backrest. Then, set the footplates parallel to the ground.

Step 4. Adjustment of legrest angle

Adjust the legrest angle with the use of locking pins which are located on the inner side of the adapter. Footrest adapters possess 3-stage adjustment of legrest angle. Adjust the footplates so that the angle between trunk and thighs is 90° (optimum sitting position is provided to the user, then).

Step 5. Protective strap

Put the protective strap on the upper part of the footrests as it prevents sliding down user's legs under the wheelchair from footplates.

Note! Do not use the wheelchair without protective strap.

On completion of the above activities, the wheelchair is ready to use.

FOLDING

The wheelchair can be re-folded in order to be transported and stored more easily.

- Before folding the frame, swing away the footplates and set them in vertical position (perpendicularly to the ground).
- To fold the frame, pull the grips in the seat upwards and lift the footrest upwards.
- To fold the backrest, press the backrest locks which are under the handgrips at the back of the backrest and pull the backrest downwards.

PUMPING THE WHEELS

- The wheelchair is equipped with pneumatic rear wheels 20'' in diameter and swivel, PU front wheels 6'' in diameter (tubeless tyre). Remember to control the air pressure in both wheels and check the condition of the tyres in search of abrasions, cuts or surface depletion.
- To balance the air pressure in the wheels, twist off the cap of the valve, insert the pump (included in the set) into the valve, and pump the air.

WHEEL LOCKS

- To lock the wheels, push the lock lever forwards until the wheel is fully blocked (separately for left and right wheel). Please, note that the wheel lock only serves to lock the wheels when the wheelchair is at a complete stop.

FOLDING BACKREST

- To fold the backrest, press the backrest locks which are located under the hand grips at the back of the backrest simultaneously and pull the backrest downwards.

ATTENDANT BRAKES

- The levers of the brakes are located behind the backrest, at the hand grips on the right and left side, and are connected with wheel lock by cable.

- In order to engage stopping lock, squeeze the lever of the brake and in order to release the brake, drop the lever. Perform this activity smoothly and with appropriate force.
- The brakes are equipped with wheel lock. To apply it, squeeze the lever of the brake and pull locking element (located at the handle of the brake).

WHEEL LOCKS ADJUSTMENT

The correct lock of the wheel lock depends on its adjustment and position. The handle of the wheel lock should be positioned so that the wheel lock shoe blocks the wheel firmly and the lever does not face excessive resistance.

- In case of excessive resistance, pull the handle forwards slightly by twisting off the locking screw which is located on the other side of the wheel lock (from the inside of the wheelchair). Set the wheel lock shoe in such position that the wheel lock blocks the wheel easily and without resistance.
- When the wheel lock shoe does not touch or block the wheel sufficiently, follow the above instruction by moving the handle in the opposite direction.

4. Technical data

Overall width	60-63 cm
Inner width	43, 46 cm
Overall depth	96 cm
Seat height from the ground	49 cm
Overall height	92 cm
Seat dimensions	42, 45 cm x 39 cm
Net weight	14.40 kg
Gross weight	16.40 kg

Maximum capacity	100 kg
Folded width	32 cm
Frame colour	black

5. Notes

Timago International Group does not bear responsibility of improper use of the wheelchair, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

Carry out the maintenance of the wheelchair components regularly:

- The upholstery and the cushion can be cleaned with damp (not wet) cloth and cold or warm water, without the use of detergents. Difficult stains can be cleaned with a sponge. Do not soak the upholstery too much. Dry away from heat sources.
- Plastic elements should be cleaned with damp cloth possibly with the use of appropriate detergents for plastic elements. In such case follow the instructions given by the producer and safety measures.










7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.
Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group
Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12
43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00
F.: +48 33 499 50 11
E.: info@timago.com



Rev. II, date 20.05.2021