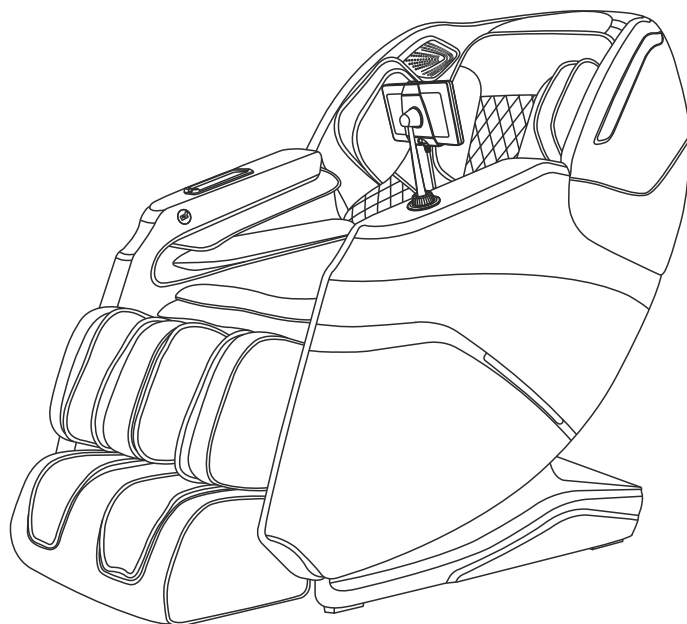
iREST SUPEARL (A336)
MASSAGE CHAIR

User manual EN

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Thanks for purchasing this appliance. Please read this manual instruction carefully in order to know how to use this appliance correctly before operation. Please pay more attention to some important safety warning and keep this manual book properly for future use.

Note: our company reserves the right to change and explain the design of the appliance. If there are individual differences between the function of the appliance circulating in the market and the description in the manual, which belongs to the normal update of the appliance, our company will not give another notice, and the specific appliance function and color shall be subject to this appliance.

CONTENTS

Safety and maintenance	02
Safety warning	04
Name and function of components	06
Function	07
Method of usage	08
Product specification	19

Safety and maintenance

1. Important Safety Warning.

- Don't allow children to touch moveable part of this product.
- Please use well-grounding power supply suited to this product.
- Please pull out the plug after use or before cleaning to avoid injury or damage to the product.
- Please operate this product according to this manual instruction.
- Don't use accessories which are not recommended.
- Don't use this product outdoors.
- Please read this manual instruction carefully before operation.
- Any other usage that is not listed in this manual instruction is forbidden.
- 20 minutes of usage each time is recommended.
- Please don't use it if the leather is broken or damaged.
- Please don't use it if the cover or leather is damaged.
- Please don't use it if the radiator hole is covered or blocked.
- Please don't drop anything into this product.
- Please don't fall asleep while using this product.
- Please don't use it if drunk or feel unwell.
- Please don't use it within one hour after having meal.
- Please don't make the massage function too strong for avoiding Injury.

2. Environment for usage.

- Please don't use it under high temperature and moist environment such as bathroom.
- Please don't use it immediately while the environmental temperature changes sharply.
- Please don't use it under heavy-dusty or caustic environment.
- Please don't use it where there is not enough space or not good ventilation.

3. People who are not suitable to use this product.

- People who are suffering from osteoporosis are not suitable to use this product.
- People who are suffering from heart disease or carrying electronic medical equipment such as heart pacemaker are not suitable to use this product.
- People who are suffering from fever are not suitable to use this product.
- Pregnant women or women that are in menses period are not suitable to use this product.
- People who got injured or skin disease are not suitable to use this product.
- Children under 14-year old or people mentally unbalanced without being Supervised are not allowed to use this machine.
- People who are told by doctor to have rest or people who feel unwell are not suitable to use this product.
- People whose bodies are wet are forbidden to use this product.
- The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.

Safety and maintenance

4. Safety.

- Check the voltage if it is suitable to the specification of this product.
- Don't use wet hand to pull the plug.
- Don't make water go inside this product to avoid electric shock or cause damage to this product.
- Don't damage the wires or change the circuit of this product.
- Don't use wet cloth to clean the electric parts such as switch and plug.
- Be away from this product under power cut state to avoid injury if the power comeback suddenly.
- Stop using this product while it works abnormal, and please consult local agent immediately.
- Stop using this product if you feel unwell, and please consult health care doctor.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.
- If the supply cord is damaged, it must be replaced by the manufacturer, its Service agent or similarly qualified persons in order to avoid a hazard.

5. Maintenance.

- This product should be maintained by certain appointed agents, users are prohibited to disassemble or maintain by themselves.
- Please don't forget to cut the power after usage.
- Don't use this product if the socket looses.
- If this product will be left unused for long, please curl up the wires and reserve this product in dry and dustless environment.
- Don't reserve this product under high temperature or near fire and avoid longtime direct sunshine.
- If the detachable cord / supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
- Please clean this product by dry cloth. Don't use thinner, benzene or alcohol.
- The mechanical components of this product are specially designed and made, it is not necessary to be specially maintained.
- Don't use edged res to thrust this product.
- Don't roll or pull this product above uneven ground, it should be lifted before moving.
- Please use it intermittently; don't make this product work continuously for long.

6. Solution of ordinary malfunctions.

- It is nature movement voice if the motor sounds while in usage.
- If the controller can't work normally, please check the plug and socket if they are connected strongly and make sure the switch is turned on.
- If the rated working time is over, the product will automatically activate the power-off button; If this product works continuously for long, the temperature protector will make the product automatically power-off, it should be used again after half an hour's rest.

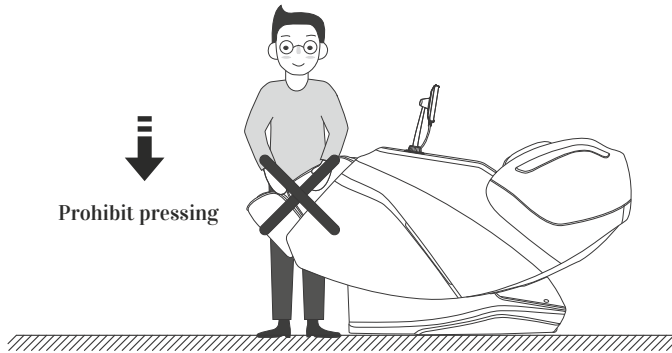
7. Storage conditions

- storage room temperature - 5°C - 35°C.
- relative humidity 20% to 80%, no corrosive gas and well ventilated room.

Use safety warning


Warning

When the footrest lifting, people should not apply pressure on the footrest to avoid the massage chair falling forward to damage the product or cause personal injuries

Warning

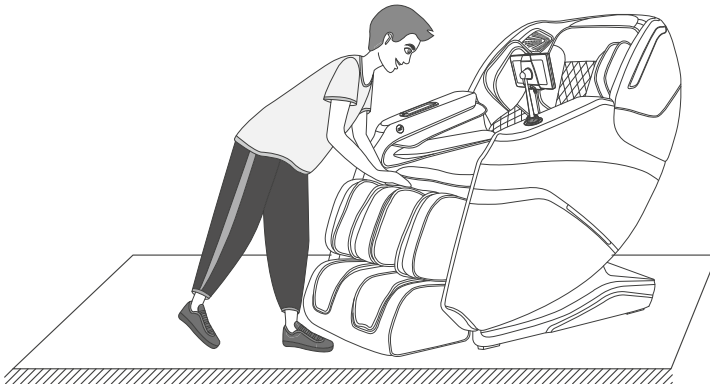
Before sitting on the massage chair, Please make sure the massage hand cannot place in the seat position, so as avoiding the massage hand to damage the product or cause personal buttocks injuries.

Use safety warning


Warning

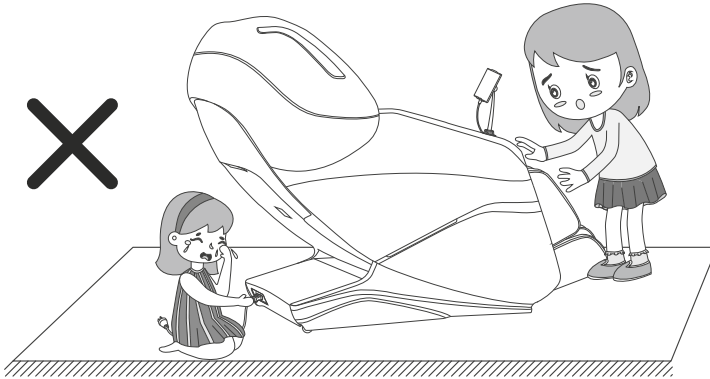Do not strive to pull out the control cable or power cord during standby or power-on state to avoid product damaged or electrocution accidents. Advise you unplug the controller cable and power cord and put away after you use the chair.



Warning

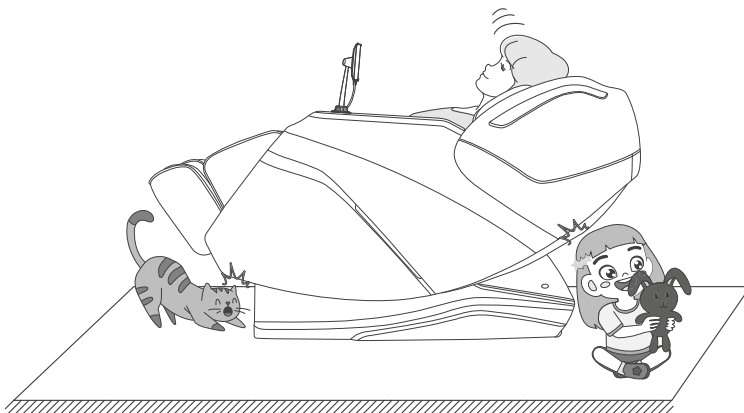When turning on the zero-gravity lying function, be sure to notice whether there are children or pets under the massage table under the bottom of the calf and the backrest to stay or play to avoid product damage and personal injury.

Name and function of components

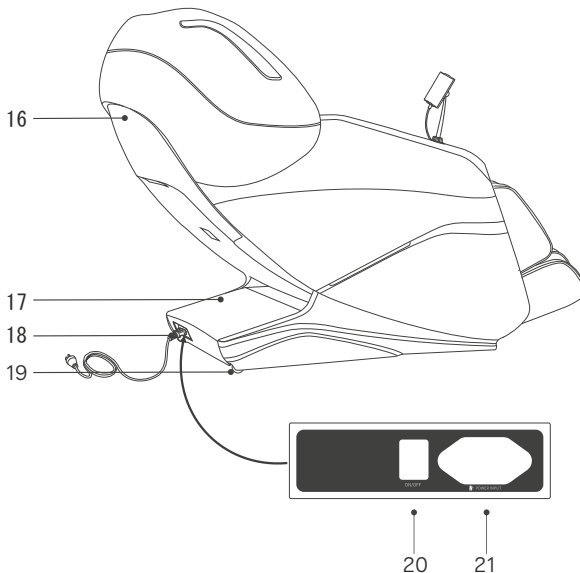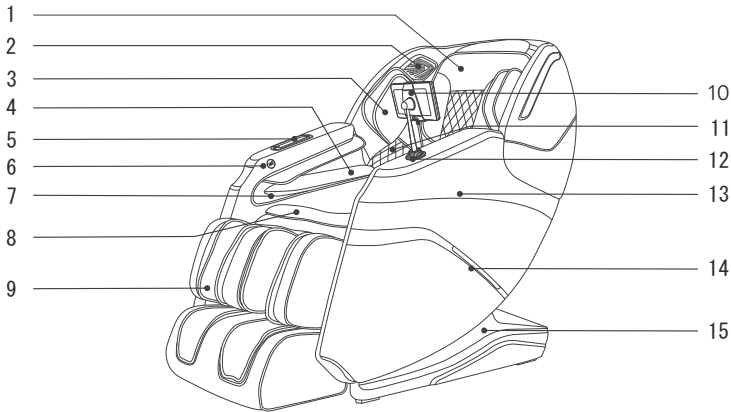
1. Pillow Pad
2. 3D digital speakers
(voice control install
In left speaker)
3. Upper arm airbag
component
4. Arm air bag componer
5. Wireless charging box
6. USB charging port
7. Sallent massage with
magnetic therapy
8. Seat cushion
9. Calf rest
10. Remote controller
11. Back cushion foot rest
12. Remote controller
bracket assembly
13. Armrest
14. LED lights
15. Side cover
16. Backrest cover
17. Drive box cover
18. Power cord and plug
19. Caster
20. Power switch
21. Power cord socket

Function

- This appliance has designed a set of massage manipulator that can walk up and down, four-wheel drive and mute design.
- There are five simulative massage techniques: finger pressing, kneading, tapping, tapping and kneading at the same time.
- Automatic shoulder detection and positioning.
- This appliance is equipped with 12 automatic massage functions, including comfort, relax, stretch, shoulder & neck, back & waist, full body massage, chiropractic, massage fatigue recovery, meridians massage, beautiful buttock massage, sleep peacefully massage and sedentary stretch massage.
- Equipped with upper body manual massage function (the whole body, fixed point and local three massage parts are selected, and the speed of each massage manipulation can be adjusted with five speed, in fixed point and local state, the manipulator can adjust the position up and down, when tapping, pressing and percussion massage, the width of the massage ball can be wide medium or narrow).
- Pneumatic massage function three modes and three levels of intensity selection. Body pressure, arm pressure, leg pressure.
- It is equipped with foot roller massage function.
- With leg frame with stepless adjustable function suitable for different height use.
- With back & calf heating function: graphene is used as far infrared heat source.
- Under the backrest frame, the backrest adopts slide rail technology to keep the zero space against the deviation wall all the time, so as to save the space to the greatest extent.
- The music player system is available, 3D digital audio systems are builtin to left and right upper arms, the bluetooth function of intelligent device can be used to connect wirelessly and play music.
- Backrest frame guide adopts curve guide technology, which completely conforms to the curve of human back, and can enjoy massage from head to thigh.
- The right armrest is equipped with wireless charging function.
- There is USB charging function on the right armrest.
- The left and right armrest are equipped with LED atmosphere lamps.
- LCD color screen controller.
- With back hyperthermia function: graphene is used as far infrared heat source.
- Shutdown automatic sitting function, manipulator automatic reset function.
- It has the function of kneading and rubbing the calf.

Method of usage

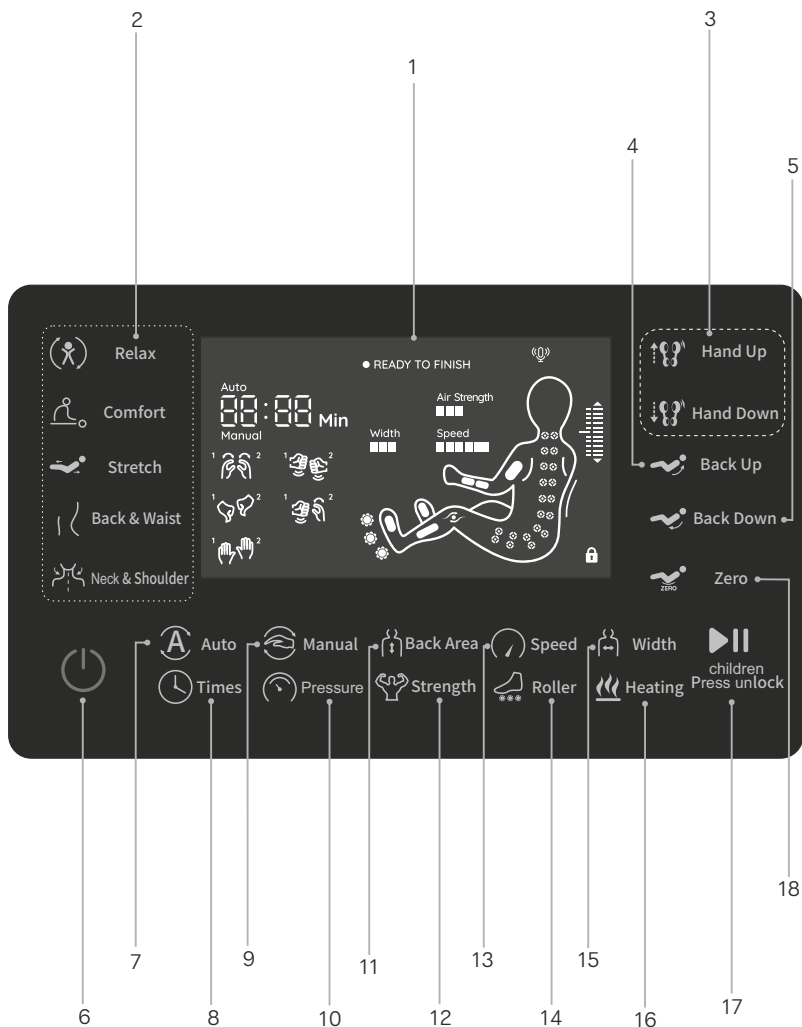LCD: display all massage functions.


Method of usage

1. LCD screen
 - Display all massage functions.
2. Auto Key
 - For details, please refer to page 11 of this manual.
3. Massage hand adjust key
 - Adjust the position of shiatsu points on shoulder massage positions adjustment (in fixed point mode and partial mode, the massage position can be adjusted).
4. Back up key
 - Adjust backrest frame to rise and calf frame down,
5. Back down key
 - Adjust backrest frame to lower.
6. Power switch key
 - Turn on or off all massage chair functions.
7. Auto Program key
 - There are 7 kinds of massage procedures, please refer to page 11 of this manual for details.
8. Times Key
 - Time adjustment please refer to page 13 of this manual for detail instructions.
9. Manual key
 - For details, please refer to page 12 of this manual
10. Air pressure area key
 - Adjust the massage area for detailed instructions, please refer to page 12 of this manual.
11. Massage area key
 - Adjust the massage area, for detailed instructions, please refer to page 12 of this manual.
12. Air pressure intensity key
 - A total of three levels of air pressure intensity can be adjusted.
13. Speed key
 - There are 5 levels of massage speed that can be adjusted.
14. Foot roller key
 - For details, please refer to page 12 of this manual.
15. Back heating key
 - A total of three levels of width can be adjusted.
16. Back heating key
 - Turn on/ Off heating,
17. Pause/start key
 - For details, please refer to page 13 of this manual.
18. Auto lying key
 - There are 3 automatizing modes, which can be switched in cycles according to your needs.

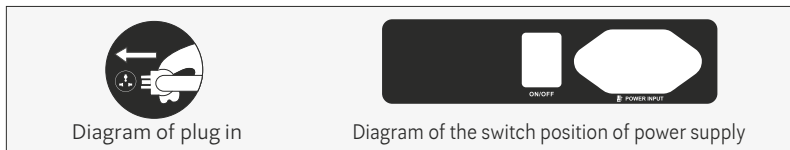Method of usage

1. Switch power supply on

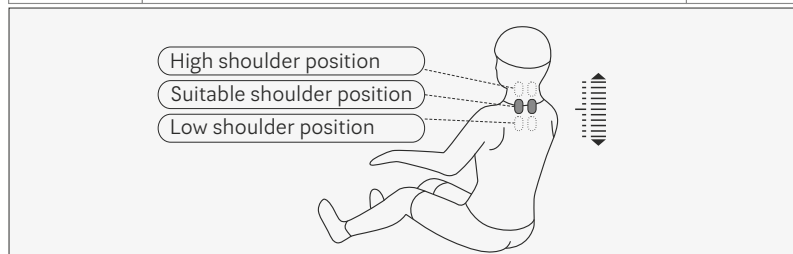
2. Start massage

- ① Press the power button to start the massage, The default is comfort massage.
- ② Dynamic adjustment.
- ③ Check the position of the moving body and shoulder to keep the sitting posture fit with the massage chair.
- ④ Fine adjustment of shoulder position. If the detected position deviates from the actual shoulder position, adjust it to the appropriate position through the up and down keys of shoulder position. The shoulder position has 10 levels in total. The shoulder position is adjusted within 10 seconds, enter the massage function

Note: in the process of starting up you can select the "Auto" function. If you repeatedly select the same "Auto" function, you can skip the starting process and directly enter the massage.

Button	Description	Display
 Hand Up	Move the massage balls upward to a suitable position	
 Hand Down	Move the massage balls downward to a suitable position	

In the process of detecting shoulder position and massage position, press up or down keys to adjust the shoulder position when there's beep sound.

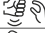
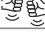
Method of usage

- Automatic massage

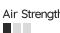
Function	Button	Description	Display	
Automatic massage	 Relax	Relax: the depth of the soft and comfortable massage, relieve body fatigue.	Auto 01	
	 Comfort	Comfort: The most comfortable massage experience.	Auto 02	
	 Stretch	Stretch: Frame and backrest frame rise and fall, and/or pressure massage, and stretch or traction the leg, remove fatigue, restore vitality.	Auto 03	
	 Back & Waist	Back&Waist: focus on back and waist massage.	Auto 04	
	 Neck & Shoulder	Neck & Shoulder: focus on shoulder and neck massage.	Auto 05	
	 Auto		Full body massage: deep massage to rejuvenate the body.	Auto 06
			Spinal massage: Focus on spine massage and fully relieve the fatigue and pain from your body.	Auto 07
			Ache relieve: Deep massage on the acupressure points can effectively relieve ache.	Auto 08
			Bottom Toing: Improve overall blood circulation of body to enhance vitality and immunity.	Auto 09
			Beautiful buttocks shaping: effective massage for the buttocks and thighs, plays a role in shaping the body, reducing the accumulation of excess body fat.	Auto 10
			Good-night sleep: massage the twelve health acupoints to help the body regulate balance, quickly improve sleep and improve sleep quality.	Auto 11
			Leisure Heath: for sedentary sitting inthe office, massage can increase the amount of physical activity and improve the body's function.	Auto 12

Method of usage

· Upper body manual massage

Function	Button	Massage ball adjustment	Display
Technique	 Manual	Shiatsu: speed, width	
		Knead: speed	
		Beat: speed, width	
		Knead pat: speed	
		Knock: speed, width	
Speed	 Speed	Speed: 5 speed adjustable	Speed 
Width	 Width	Width: The width of the third gear can be adjusted	Width 
Back stretch	 Back Area	Fixed position and spot massage	H1 Manual
		Small area local round trip massage	H2 Manual
		A round trip massage	H3 Manual
	 Hand Up	Massage position adjustment (in fixed point and local mode, massage position can be adjusted)	Manipulator uplink
 Hand Down	Manipulator descending		

· Pneumatic massage

Function	Button	Description	Display
Air pressure Mode	 Pressure	Full pressure: air pressure massage on arms	
		Arm: air pressure massage with the arm as the focus	
		Leg: barometric massage for leg, foot	
Intensity	 Strength	Adjustable in three levels	Air Strength 

· Foot roller massage


Function	Button	Description	Display
Foot roller	 Roller	Press this button to turn on or off the foot roller massage, and has a weak, medium, strong three levels of intensity can be adjusted to close the four states of cycle switching	
Calf kneading	 Roller	Long press „Sole roller“ for 3 sec, to turn on/off the calf kneading function	

Method of usage

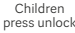· Heated massage

Function	Button	Description	Display
Heat	 Heating	Press this button to turn on or off the back far infrared hyperthermia. Turn on 3 minutes later have a warm.	

· Pause/start massage button

Function	Button	Description	Display
Pause/Start		Press this button in the process of massage, and all massage movements will be suspended urgently when time continues to countdown), and then press this key to resume all before suspensionmassage.	
V-chip	 Children press unlock	Press and hold Pause/start key for 3 seconds, display shows  that lock is successful, and other keys cannot be used after being locked; press and hold Pause/start for 3 seconds in locked state, and the display disappears  indicating that unlock is successful and keys function returns to normal.	

· Adjust lying Angle

Function	Button	Description
Sit up	 Back Up	Press this button, backrest frame slowly rise, calf frame slowly fall, and issued „di...di..." until it reached the top. Press this button again in the process of sitting, then stop sitting immediately.
Lay down key	 Back Down	Press this key, backrest frame slowly down, calf frame slowly up, and issued "di...di..." until it stops. Press this key again in lying down process, stop lying down immediately.
Lie down automatically	 Zero	Press the button every time to switch one automatic lying mode. There are three modes in total.

· Other manual functions

Function	Button	Description	Display
Timing	 Times	Time adjustment, every time to press will add 5 minutes, can not exceed 30 minutes, if more than 30 minutes return to the minimum time value.	<i>20min</i>

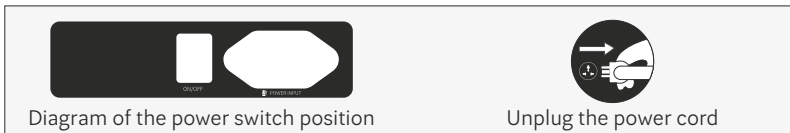
Method of usage

3. Operation Instructions for Bluetooth Speaker Play

1. Enter the smart device main menu, Open "Settings" and select the „Bluetooth" options. Open "Bluetooth", and Bluetooth will automatically „Search for a New Bluetooth Device".
2. The smart device will search for the corresponding Bluetooth model (e.g. IMCM-XXX), Click he "Match" button, if you are asked whether to match with the device, you just select "Match" to complete matching.
3. Turn on "Music Player" in the smart device and select the song to play.

4. Turn off the power and stop massage

- Press power key to turn off all massage functions when in massage process, the chair will reset automatically. All the massage functions are off while the chair will not reset if the massage time is over.
- Cut off the power of the whole machine, Figure(switch power off)

5. Voice Control Mode

- When the massage chair is turned on or in standby mode the sound "Hi Alice" or "Hey Alice" will be produced to enable the voice control function. Alternatively, you can press the voice control button on the armrest to enable the voice control function.
- Press and hold the zero gravity key of the manual controller for 3 seconds to turn on or off the device control function.
- After such function is enabled successfully, the massage chair will reply „I am here". At this moment, once a corresponding voice control entry to be implemented is given within six seconds, the corresponding voice control entry to be implemented is given within six seconds, the corresponding voice control function can be performed after a reply is received from the massage chair if no correct voice control entry is given within six seconds, the voice control function automatically will be off automatically and needs to be enabled again.

Noted: When waking up the voice control function, please use English, the speech rate is moderate, Voice commands should be follow up the voice entry.

Method of usage

Voice Control Entry


Entry: "Hi Alice" or "Hey Alice". Answer: I am here.

No.	Voice control entry	Answer entry
1	Massage on	Answer: Ok, massage on
2	Massage close	Answer: Ok, massage close
3	Comfort Massage	Answer: Ok, comfort massage
4	Relax massage	Answer: Ok, relax massage
5	Full body massage	Answer: Ok, full body massage
6	Neck and shoulder massage	Answer: Ok, neck and shoulder massage
7	Back and waist massage	Answer: Ok, back and waist massage
8	Stretch massage	Answer: Ok, stretch massage
9	Open the air pressure	Answer: Ok, open the air pressure
10	Close the air pressure	Answer: Ok, close the air pressure
11	Up the seat position	Answer: Ok, up the seat position
12	Down the seat position	Answer: Ok, down the seat position
13	Change the other mode	Answer: OK, change the other mode
14	Go little down	Answer: OK, go little down
15	Go little up	Answer: Ok, go little up


Method of usage

6. The calf installation.

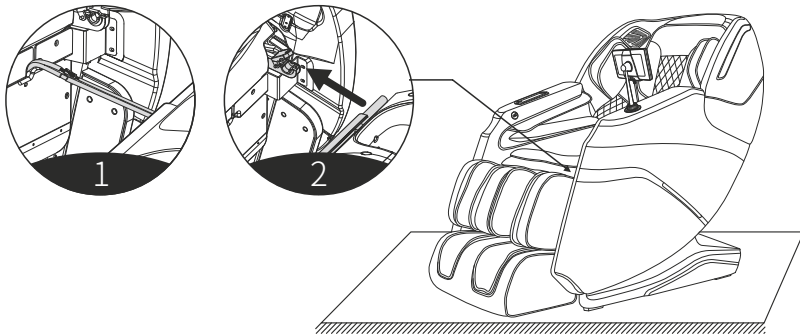- After connecting the wires and the air hose between the chair seat and the leg rest, unfasten the fixed pipe belt and insert the connecting part into the seat frame. (figure 1)
- Place the round tube from calf left and right sides to the legs brackets. (figure 2)

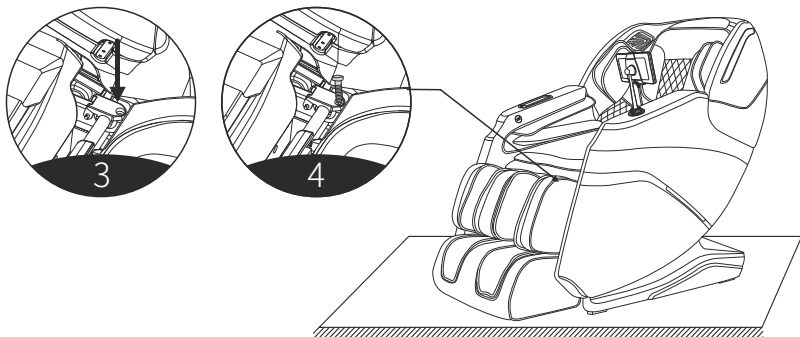
- Then turn the legs fixing cover over the round tube on both sides of the legs. (figure 3)
- Then use 2pcs M4 screws fix the legs on the massage chair, tum up and down to check if there are any problems, the wires and air hose get stuck or not, finish the installation. (figure 4)

Method of usage


7. Instructions for removal and use of head cushion and backrest

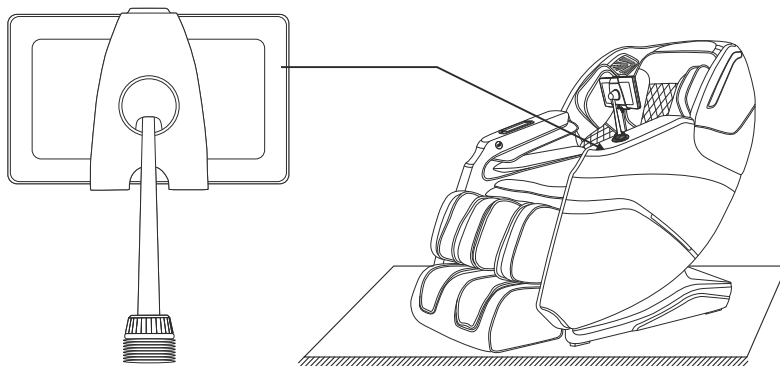- After matching the thread on knob of remote controller bracket assembly support to the thread position of support mounting seat on arm rest of massage chair, turn it clockwise and tighten it.

8. Instructions for removal and use of head cushion and backrest

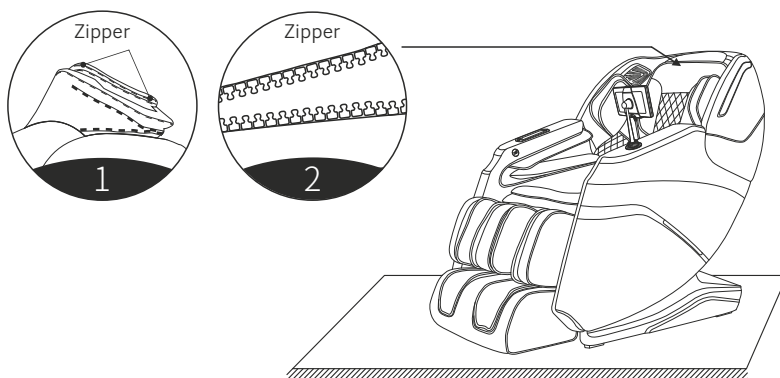
- The use of the head cushion can reduce the intensity of kneading massage on the neck and shoulders. You can decide whether to use the head cushion according to your own needs (the head cushion is recommended). The backrest and the back cushion are connected by a zipper (1). The head cushion and the backrest are connected by a zipper (2).

Method of usage

9. Floor Protection

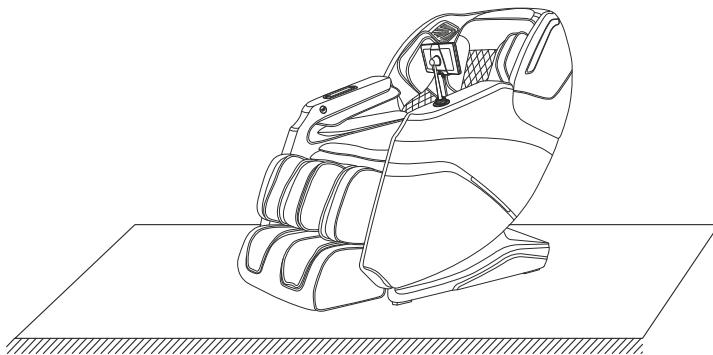
- Put the heavy massage chairs on the wood floor for a may damage the floor, to please place a carpet or other object to avoid this damage.

Caution: Do not move the chairs with its wheels on wood floor, rough ground or in a narrow space. Two people are required to lift the chair. During the lifting period, take the calf rest end sealing plate and backrest cover grooves as the force-bearing point (Please do not lift armrest directly.)

10. Method of movement

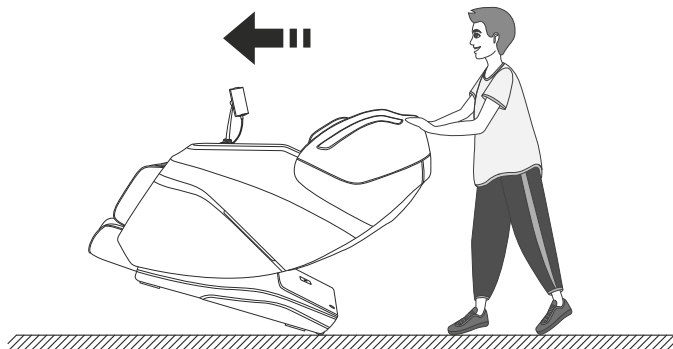- Make sure that all wires are far above the ground, tit the backrest backward to a certain degree (with the gravity center resting on the wheels), push the chair forward or backward with your hands and finally resume the chair to the normal position in a slow and gentle manner. (Note: During movement you must turn off the power first and then unplug the power cord and the controller cord.)

Warning: Massage chair rear wheels are only suitable for short distance movement for long distances, pls use other assistive tools to move the massage chair.

Product specification

Model: iRest SL-A336

Configure: 3

Name: Massage Chair

Rated voltage: 110-120V~ 60Hz

220-240V~ 50Hz/60Hz

Rated power input: 140W

Rated time: 20Min

Safety structure: Class I

Material introduction: PVC, PA, steel parts and electric & electron parts

MATERIAL	PU	LEATHER	CLOTH	WOOD

Bluetooth frequency range and power of 2.5 mW

Manufacturer's declaration: Product is compliant with Directive: 2014/53/EU.

Full text of declaration of conformity is available online

<https://restlords.com/en/declaration>

SAVE THESE INSTRUCTIONS

Dear user, you can contribute to environmental protection.

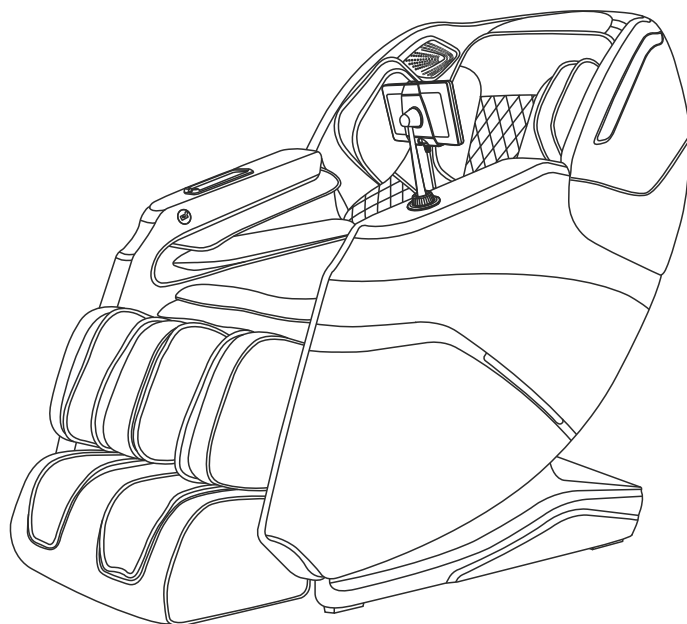
Be environmentally responsible and return your used equipment to the relevant collection point designated by your local administration. The materials used to manufacture the device may be valuable recyclable resources and, if disposed of incorrectly, can pose a risk to the natural environment.

Remember - do not dispose of the device together with your household waste.

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