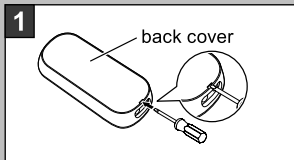
Pain Reliever HeatTens (HV-F311)

Disposal of the device and battery

Before disposing the main unit, remove the rechargeable battery and please follow local regulations on environmentally safe disposal of battery and main unit.


- Make sure to remove cord with pads and AC adapter from the main unit before disassembling.
- Make sure to wear out the battery and the  icon displays.
- Make sure nothing is shown on the display afterwards when pressing the power button.
- Take care to avoid cuts from sharp edges.


1 Remove the back cover of the main unit.

Insert a flat head screwdriver into the slot to open the back cover.

Note: Take care to avoid injury while opening the cover.

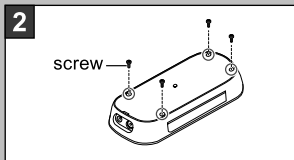
2 Remove the four screws on the back side.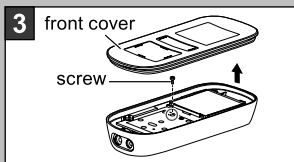

3 Turn over to the front.


Remove the front cover and the screw attached inside.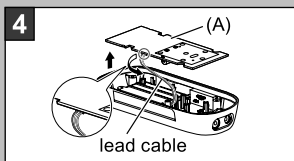

4 Lift the circuit board (A) and remove the connector of the lead cable from (A).

Twist the connector to remove.

Note: Do not cut the lead cable by scissors etc. Otherwise it may cause short-circuiting and result in a fire.

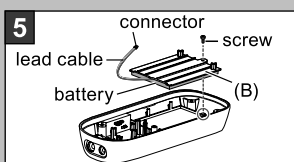5 Remove the screw and the battery compartment (B).

Remove the tape that holds the lead cables.
Do not remove the adhered battery from (B).

Note: To prevent the removed battery from short-circuiting, insulate the battery by covering the connector with tape.

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