U-Tech. Ref F833

Manual de instrucciones  
Instruction manual  
Mode d'emploi  
Bedienungsanleitung  
Manuale d'istruzioni

Masaje Vibratorio Corporal  
Vibrating Body Massage  
Massage Vibratoire Corporel  
Körpermassagegerät  
Massaggio Vibratorio Corporale

---

## **Advertencia**

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## **Warning**

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

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## EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU.
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU.

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## Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

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### **Welcome!**

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

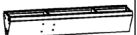
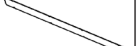
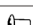
The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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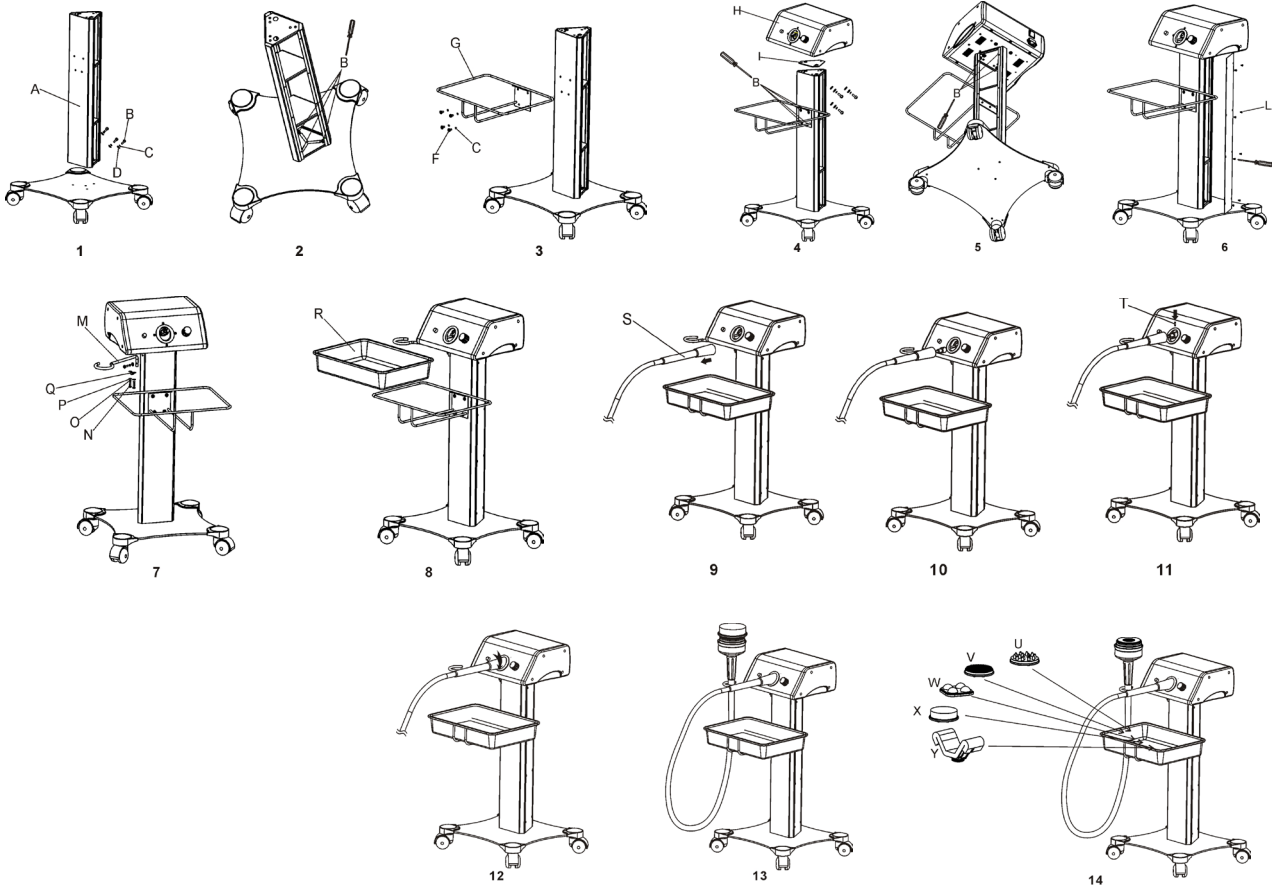
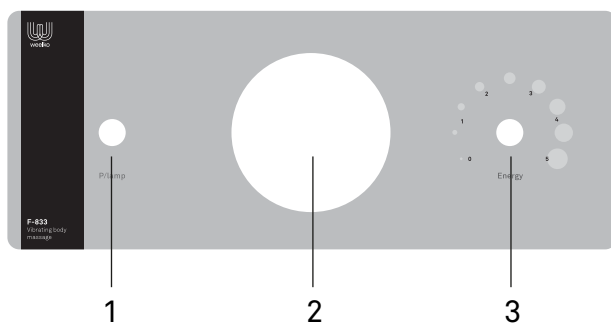
---

### REVOLVING SPEED OPTION

1. The first gear is used for the small parts of the body, such as the knuckles, the neck, or the sole of the feet.
2. The second gear is used to mobilize the grease and to remove wrinkles.
3. From the third to the fifth gears, they are used to massage and relax the muscles. These three gears can be adapted according to the way we apply the massage.

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---

## MASSAGE PROBE OPTION

1. (U) Thick massage probe stimulates the muscles, just like the physiotherapist uses his finger to massage your body, it helps to ease the pain, to remove fatigue and relax your muscles and the hard fat. It can massage the fat in order to create heating once the speed gets to 3-5, this is a good way to boost weight loss.
2. (V) Thin massage probe is mainly helpful used on rough skin, wrinkles and soft fat; it can help you to lose fat massaging the skin and the adipose tissues. We suggest using the probe with a low gear 2-3.
3. (W) Square massage probe is mainly used on larger parts of the body, such as your back, it's helpful to massage the body when there's a lack of exercise. It can prevent your muscle from fatigue and stiffness, and it also helps to keep joint flexibility, to make your body softer and to consume the fat to contribute to the weight loss.
4. (X) It can be applied on all acupuncture points of your body, covers every parts of your body, palm and soles of feet. We suggest using 1 gear when you want to do massage on the acupuncture points (it is better to wrap the probe before applying on your body)
5. (Y) It is mainly applied on the arms, neck, side, and thigh; when doing massage, it can effectively relieve muscle tension and fatigue. We suggest using 3-5 gear under that conditions (it is better to wrap the probe before applying on your body)

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## Cautions

- Do not hook or place the massage probe on ground after starting the host machine. When using the machine, the operator must hold the handle of massage probe.
- Do not replace or remove the massage probe.
- When using the machine, please pay attention to the range of operation, and do not pull the soft tube over excessively.
- The soft tube cannot be bended excessively, for example, the camber cannot over 100°, when holding part cannot be less than 150°.
- Do not apply lubricant oil on axis and motor.
- Host machine and massage probe cannot be demounted or replace or repair; when the machine is under abnormal conditions, please turn to your distributor for help or ask the professional engineer to repair, you cannot open the machine or repair it by yourself.
- Please follow up the instruction from doctor when you get ill.
- Please do not use the machine with the power supply and plug damaged, or collided, or got rain, instead you should send the machine back to the service center to get examination or maintenance.
- It is not allowed using machine under the damp environment, such as bathroom, etc.

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## FAQ

The probe suddenly stops emitting:

- If the probe is separated from the skin, it will stop emitting after 5 seconds. This is a safety feature and should not be mistaken for a system fault.
- Touch the skin with the probe and it will start emitting again after 5 seconds.

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## **Maintenance**

- Any oil on the probe must be removed immediately.
- After use, please clean the probe and keep it probe. Store it in the probe-protecting sleeve at all time.
- Store the probe carefully to avoid damaging it.

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## TECHNICAL SHEET

Weights and sizes	Packing size (cm)	83,5x43,5x39,5
	Gross weight (kg)	18
	Net weight (kg)	15
Working power	Level 1/2/3/4/5	800 / 1600 / 2400 / 3200 / 4000 RPM
Electrical characteristics	Rated voltage	100V ~ 240V
	Frequency	50Hz/60Hz
	Rated power	100 W

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