ES Manual de instrucciones
EN Instruction manual
FR Mode d'emploi
DE Bedienungsanleitung
IT Manuale d'istruzioni

WKP006

WIND

MICROMOTOR PARA PODOLOGÍA
MICROMOTOR FOR PODIATRY
MICROMOTEUR PODOLOGIE
Fußpflegegerät ideal für Podologie
MICROMOTORE PER PODOLOGIA

USO DEL MANUAL

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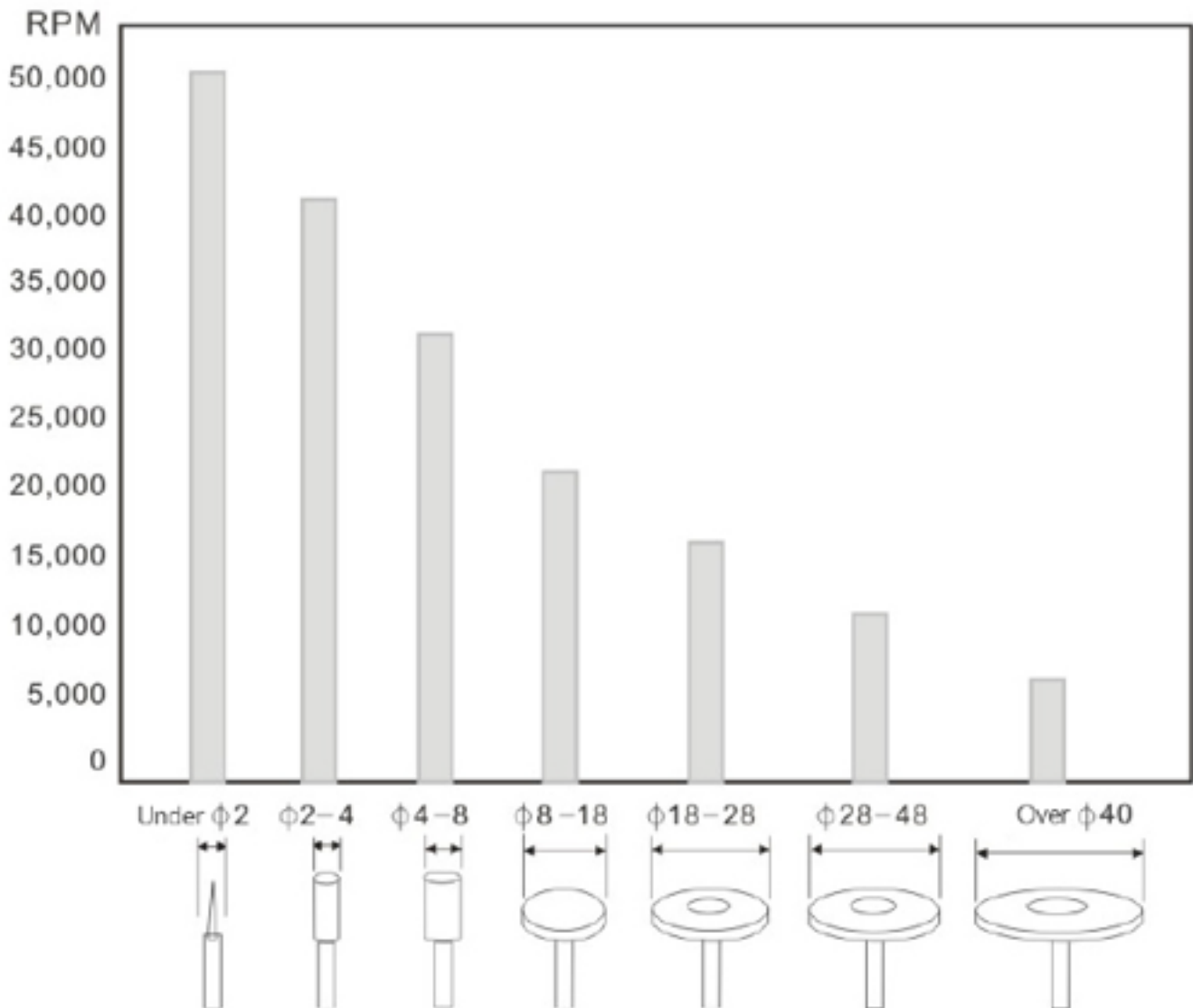
FICHA TÉCNICA

USE OF THE MANUAL

It is important that you read and retain this user manual for future reference. It contains important information on its use and maintenance.

You can also ask a professional for guidance to make sure you are using the bed correctly.

WARNING

- Never under any circumstances attempt to open or inspect the internal components of the device. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the device in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards or contact the dealer for service.

EC-DECLARATION OF COMPLIANCE

This product meets the following directives:

- Low Voltage Directive: 2014/35/EU
- Electromagnetic Compatibility Directive: 2014/30/EU

RECYCLING

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve natural resources and improve standards of environmental protection regarding the treatment of electrical waste.

WELKOME!

Thank you very much for your purchase. Please read carefully the instructions before using the device. We accept no responsibility for damages caused by misuse.

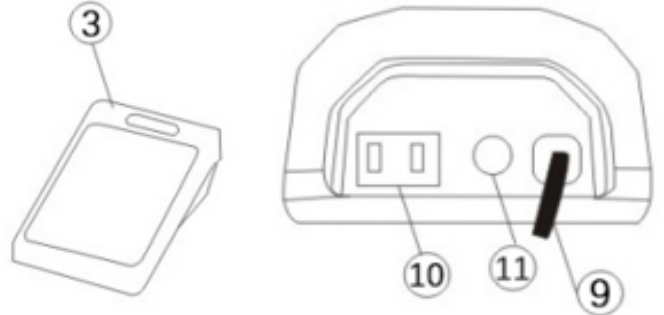
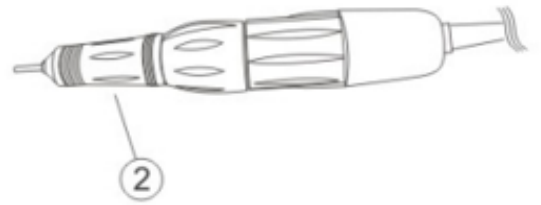
The company reserves the right to amend the details of the devices with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

CONTENT

Please check that there are no missing parts when you receive the parcel:

- Micromotor, handpiece and footswitch.

PARTS OF THE EQUIPMENT

1. Main unit
2. Handpiece
3. Footswitch to adjust speed and to switch the device on and off
4. Motor connector
5. Speed control
6. Direction of rotation control
7. Power supply connector
8. Power light
9. Footswitch connector
10. Fuse

OPERATION INSTRUCTIONS

1. Please make sure the main unit is switched off before plugging it into a wall socket.
2. Connect the handpiece to the main unit and place the handpiece on its holder carefully.
3. Make sure the speed position is 0 before switching the device on.
4. Make sure the drill is securely fastened and that it cannot drop during treatment.
5. Hold the handpiece with one hand and switch the device on with the other hand.

OPERATION INSTRUCTIONS

EN

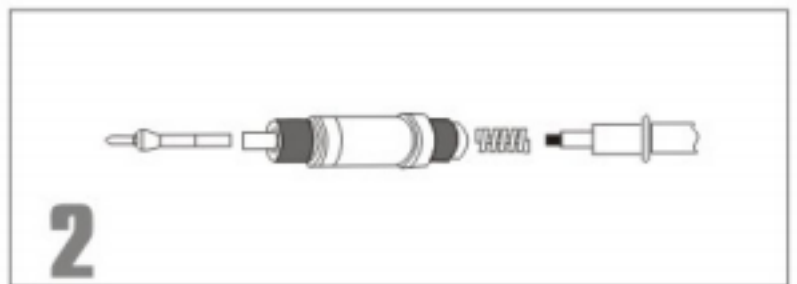
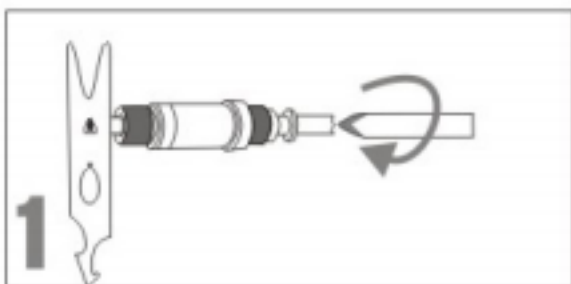
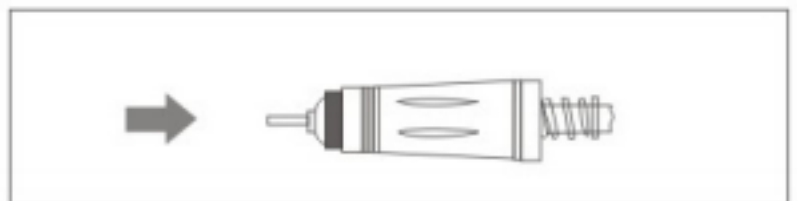
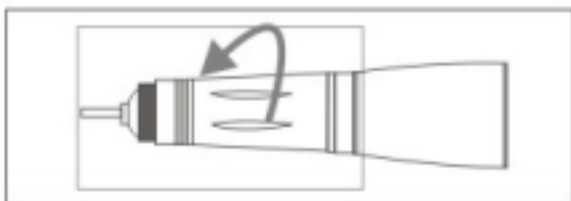
6. Check the handpiece direction of rotation when switching the device on.
7. Switch the device off with the footswitch to check if it is working properly.
8. Use Left/Right to select the direction of rotation to the left or to the right. Switch the device on to check if both directions are working properly.
9. Once the device is well configured, the treatment can be started.
10. To change the drill, turn the handpiece as shown in the pictures below with the help of the part supplied to remove it.

* Handle the handpiece with care. Drops or impacts could damage the internal components and affect its functioning. Should they occur, please check first the functioning of the device at minimum speed.

* If the engine stops suddenly, check the fuse and avoid electric overload.

* Keep the handpiece always with a mounted drill and make sure it is blocked.

* Use only compatible accessories with this device. Follow the recommendations made by the drill manufacturer regarding caution and recommended speed.

CAUTIONS

- * Do not use a rusted or twisted drill.
- * Gradually increase the device speed.
- * Never change the drill when the handpiece is working. The drill chuck must be opened only when the handpiece is completely stopped.

* Use task-specific clothing and equipment, including safety glasses and face shield visor against fine dust. Use a powerful vacuum cleaner to prevent dust buildup.

* Keep your workplace clean and free from other objects to work under good conditions.


MAINTENANCE

EN

1. Clean the handpiece with a soft cloth soaked in alcohol and dry it well after disinfection. Make sure the alcohol does not enter inside the device.
2. Do not submerge the handpiece into water or oil. The internal components could be damaged.
3. Use a compressed air gun to remove traces of dust. Use it carefully to prevent the compressed air from entering inside the handpiece and damaging its internal components.

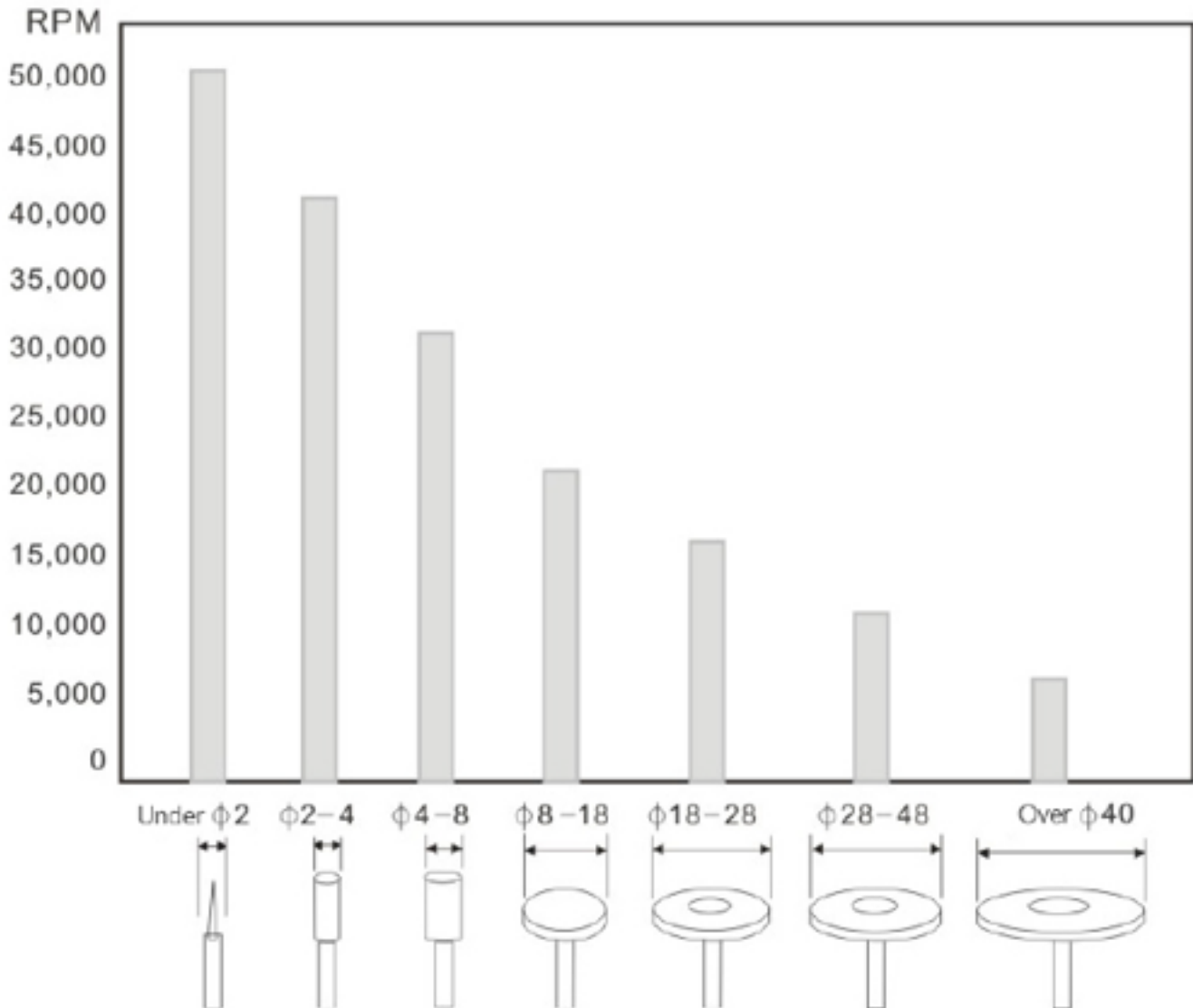Product size	13,5 x 11,5 x 8 cm
Handpiece dimensions	16,5 x 3 cm
Weight	1.7 kg
Torque	2,8 N/cm
Input voltage	220V/230V/240V
Speed	35.000 rpm
Maximum input current	2A
Power	65W
Compatibility	2.35mm, 3.0mm and 3.175mm drills (ISO 7785-2 TYPE 2). Minimum and maximum length of the drill: 10/35 mm.

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