ES Manual de instrucciones  
EN Instruction manual  
FR Mode d'emploi  
DE Bedienungsanleitung  
IT Manuale d'istruzioni

**P03**

## **MOVE**

PEELING FACIAL PORTÁTIL  
PORTABLE SKIN SCRUBBER  
PEELING PORTABLE  
TRAGBARER SPATEL  
PEELING FACIALE PORTATILE

ES

EN

FR

DE

IT

## USO DEL MANUAL

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---

## USE OF THE MANUAL

It is important that you read and retain this user manual for future reference. It contains important information on its use and maintenance.

You can also ask a professional for guidance to make sure you are using the bed correctly.

---

## WARNING

- Never under any circumstances attempt to open or inspect the internal components of the device. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
  - Never use the device in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components.
  - Never use the unit outdoors; exposure to the elements can cause damage.
  - If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards or contact the dealer for service.
- 

## EC-DECLARATION OF COMPLIANCE

This product meets the following directives:

- Low Voltage Directive: 2014/35/EU
- Electromagnetic Compatibility Directive: 2014/30/EU

---

## RECYCLING

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve natural resources and improve standards of environmental protection regarding the treatment of electrical waste.

---

## WELCOME!

Thank you very much for your purchase. Please read carefully the instructions before using the device. We accept no responsibility for damages caused by misuse.

The company reserves the right to amend the details of the devices with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

## CONTENT

On arrival of the beauty bed, check the presence of the following items:

- Ultrasound peeling spatula
- Power cord

## GENERAL INFORMATION

The Portable Skin Scrubber is a beauty instrument designed to improve skin problems. It uses an ultrasonic vibration with a frequency of 25-27 KHz, which removes toxins, blackheads, acne and dead skin particles.

It is well recognized by customers that this is an easy-to-use instrument which clearly improves a wide variety of skin problems. It works on low voltage which allows to reduce melanin and pore size, helps skin sterilization, pore deep cleansing, helps reduce wrinkles and whitens the skin.

A 45-degree angle operation should be adopted to accelerate the metabolism of epidermic cells and enhance the shedding of the ageing cells, while softening keratin and restoring the skin elasticity in order to give the skin a firm, smooth and delicate appearance.

## FUNCTIONS

1. Treats skin ageing, removes the excess of fat and the dirt hidden in skin pores.
2. Invigorates skin and improves its appearance.
3. Cleans skin deeply and clarifies the skin without pain.
4. Dilutes pigmentation. It's ideal for acne-prone skins.
5. Reduces the pore size.
6. Whitens the skin and makes it shine.
7. Smooths wrinkles. Increases the elasticity of the skin.
8. Digital timer for 15-minute treatment. The device stops automatically after the selected time is over.
9. Portable instrument.

## PRODUCT DESCRIPTION

## Operating instructions

1. Plug the power cable into a wall socket and plug the adapter into the spatula.
2. Press the On/Off button to turn the device on. When you turn on the instrument, the continuous wave and the lowest power are selected by default.
3. Choose the most appropriate wave for each treatment. There are three types of waves: pulsed wave, continuous wave and combined wave (it combines the pulsed and continuous waves). If the continuous wave is chosen, the indicator will not change. If the pulsed wave is chosen, the indicator will flash. If the combined wave is chosen, the indicator will light during one second and then it will flash. The type of wave can be changed depending on the treatment. The indicator and its color change depending on the selected power, so the indicator shows the power and the flashing shows the type of wave.
4. Adjust the intensity accordingly. There are three power levels: low, middle and high. Low power is established by default.
5. The treatment time by default is 15 minutes. The device will beep after the selected time is over and it will switch off automatically after this period of time.

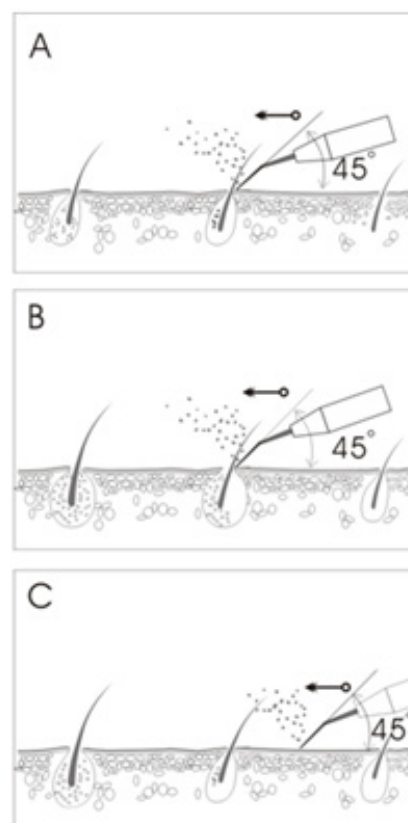
1. Turn the device on. The indicator will light up and the spatula will start working. The continuous wave and the lowest power are selected by default.

2. Adjust the type of wave and the power according to the type of treatment to be made. The combined wave is ideal for sensitive skins.

3. Clean the area to be treated with a lotion.

4. Place the spatula on the skin with an inclination of 45 degrees, as shown in the pictures below. Move the spatula forward deeply to clean the area to be treated. Choose the power according to the characteristics of the treatment. (Note: Clean the skin and the spatula with clean water. Keep the skin moistened during the treatment.)

\*\*Please use the spatula as shown in pictures A, B and C.

---

## INSTRUCTIONS

- A. Acne removal without pain. The acne on the skin is atomized by ultrasonic vibrations in a very short time. So acne can be removed when cleaning the skin.
- B. Redness removal. Redness and makeup residues are removed by ultrasonic vibration at a frequency of 25K.
- C. Skin cleaning. The spatula is used to clean dirt from the skin surface and to improve skin ageing. By means of its effective vibration, cleaning effects are really visible.
5. Clean the skin with water after the treatment.
6. Clean the spatula and sterilize it with alcohol.

---

## NOTES

1. It is ideal for patients with dry or oily skin. The skin should be revigorated after the treatment.
2. Please apply water, a lotion or gel on the skin before the ultrasonic treatment.
3. The continuous wave is the best option to clean the skin or treat skin ageing. Pulsed wave is perfect for massaging the skin and smoothing wrinkles.
4. Keep the spatula clean and sterilize it before the next treatment.

5. Do not use it around the eyes or on injuries.
6. Do not stay for too long on the same position to avoid damaging the skin. The spatula should be moving all the time.
7. Do not use the spatula if it is damaged or deformed.
8. Please treat the skin gently and use the spatula inclined at a 45-degree angle.
9. The spatula grinds a little when working. This noise is caused by the frequency of vibration.
10. Do not use this device in children, pregnant women or people with psychopathologies.
11. Do not use this device if the cable is damaged or the product has dropped into water. In this case, it should be returned to the manufacturer to be checked and repaired.
12. Do not use this device outdoors.
13. Do not use this device in damp places, such as bathrooms.
14. Do not use this device with a different cable because its parameters may not match.
15. Do not use this device with patients suffering from skin diseases or sensitive to ultrasonic stimulation.
16. Keep it turned off if not used.
17. Please contact the manufacturer if the cable is damaged to be replaced. To avoid accidents, do not attempt to repair the machine yourself.

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