ES Manual de instrucciones
EN Instruction manual
FR Mode d'emploi
DE Bedienungsanleitung
IT Manuale d'istruzioni

WKE004

FINEX

BAÑO DE PARAFINA
LUXURY PARAFFIN BATH
CHAUFFE-PARAFFINE DE LUXE
LUXUS PARAFFINBAD
SCALDAPARAFFINA

USO DEL MANUAL

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USE OF THE MANUAL

It is important that you read and retain this user manual for future reference. It contains important information on its use and maintenance.

You can also ask a professional for guidance to make sure you are using the bed correctly.

WARNING

- Never under any circumstances attempt to open or inspect the internal components of the device. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the device in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards or contact the dealer for service.

EC-DECLARATION OF COMPLIANCE

This product meets the following directives:

- Low Voltage Directive: 2014/35/EU
- Electromagnetic Compatibility Directive: 2014/30/EU

RECYCLING

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve natural resources and improve standards of environmental protection regarding the treatment of electrical waste.

WELKOME!

Thank you very much for your purchase. Please read carefully the instructions before using the device. We accept no responsibility for damages caused by misuse.

The company reserves the right to amend the details of the devices with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

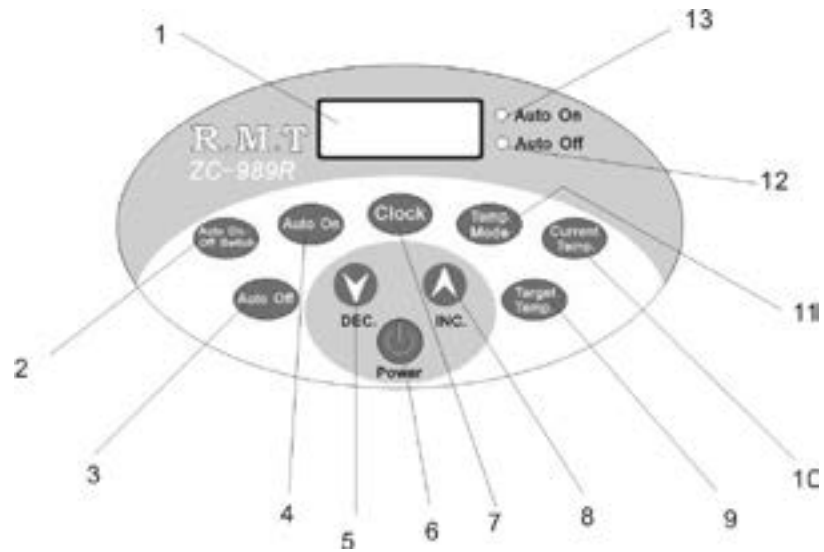
CONTENT

On arrival of the package, check the presence of the following items:

- Luxury paraffin bath

EN

GENERAL INFORMATION

The paraffin bath is popular for beauty and home treatments because of its outstanding effect on dry and aged skins, but also on anti-wrinkle treatments.

This easy-to-use equipment is popular among the beauty centres all over the world for the following features: a temperature controller and display, a quick melting, a timing controller and display, a temperature switch in Fahrenheit and Celsius degrees and a large capacity.

GENERAL DISPLAY

1. Display
2. Auto on/off switch : Automatic mode switch
3. Auto off: Time setting to end the automatic mode
4. Auto on: Time setting to start the automatic mode
5. DEC.: Temperature/time down
6. Power button
7. Clock
8. INC.: Temperature/time up
9. Target temperature
10. Current temperature
11. Temperature unit setting
12. Automatic mode off indicator
13. Automatic mode on indicator

OPERATION

1. To switch the equipment on, press the button "Power".
2. The first time you use the equipment, the clock needs to be adjusted on time. To do so, press the button "7" and the first two numbers of the clock will shine. Then, by pressing the buttons "5" and "8", you will be able to adjust the hour. If you press the button "7" again, the last two numbers of the clock will shine. Then, press the buttons "5" and "8" to adjust the minutes. To complete the clock setting, press the button "7" again.
3. To adjust the working mode, press the "2" button. Then, the "Auto on" and "Auto off" indicator will shine. If you press the "4" button, the numbers of the display will shine and you will be able to adjust the time at which the automatic mode has to be switched on by pressing the "5" and "8" buttons. If you press the "3" button, you will be able to set the time at which the automatic mode has to be switched off. If the indicators ("12" and "13") do not shine, it means that the machine is working in the manual mode.
4. Temperature unit setting. Press the "11" button to select the temperature unit. Then, the "5" and "8" will allow you to choose either the Fahrenheit (F) or the Celsius (C) scale.
5. Target temperature setting. Press the "9" button to set the target temperature. The "5" and "8" buttons will allow you to reach the temperature needed.
6. Current temperature display. Press the "10" button to see the current temperature.

REMARKS

- After turning on the automatic mode, do not unplug the equipment. Make sure that the clock is on time.
- If the automatic mode is not used, make sure that you switch the equipment off after use.
- The machine can save the temperature that was set last time.
- Before starting the treatment, check the temperature on your hand to see if it is the proper one and make sure it will not be uncomfortable for the customer.
- If the skin is damaged, do not apply this treatment.
- After finishing the treatment, do not pour the remaining paraffin when it is still liquid. Turn off the unit and wait for the paraffin to get colder and become completely solid. Then, turn the equipment on again and wait for the paraffin at the bottom to be melted and you will be able to remove the paraffin block easily.
- In order to shorten the melting time, choose the maximum temperature. Once the paraffin is completely melted, then adjust the temperature at the proper one.
- If the power cord is damaged, it should be replaced by one provided by the manufacturer or the authorised technical service.

TECHNICAL SPECIFICATIONS

MELTING TIME	30 minutes
TEMPERATURE RANGE	46 - 60°
CAPACITY	5000 ml
RATED POWER	200W
RATED VOLTAGE	AC 220V ~ 240V

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