ES
WKB001
Manual de instrucciones
WE-2020-WKB001-1

PRESOTERAPIA

ADVERTENCIA

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PRESSOTHERAPY

WARNING

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms, etc. to prevent damage to the electrical components of your unit.

- Never use the unit outdoors; exposure to the elements can cause damage.

- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

- Do not use on pregnant women, on people with pace-makers (any electrical or rhythmic heart-regulating device) or with skin infections.

RECYCLING

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to



incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

PACKAGE CONTENTS

- These components are subject to change without prior notice.
- Appearance of product may differ from the image.

No.	PRODUCT	QTY.
1	RIGHT LEG	1
2	LEFT LEG	1
3	RIGHT ARM	1
4	LEFT ARM	1
5	RIGHT FOOT	1
6	LEFT FOOT	1
7	WAIST	1
8	CABLE BLANKET	1
9	TUBES	1
10	MAIN UNIT	-
11	FEEDER CABLE	-

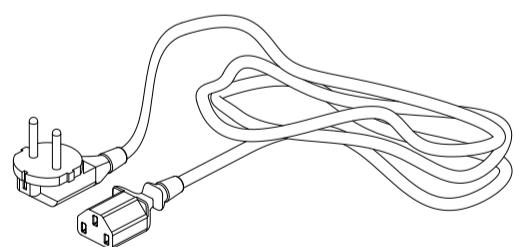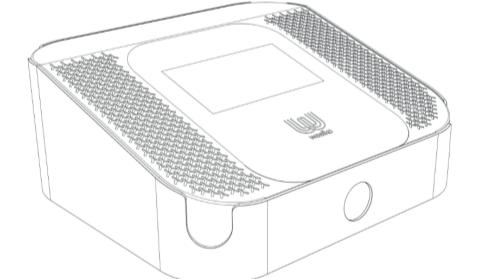INSTRUCTIONS OF USE

- Connect the power cord via the rear panel.
- Connect the tubes properly to the rear panel. Follow the numerical instructions.
- Place the blanket with the cables on a flat surface so that the patient can lay down on it. Cover the blanket with disposable paper to ensure the best hygienic conditions.
- Wrap the suit around the user's body. Remember to use a disposable garment underneath the pressotherapy suit.
- Turn on the power button on the front panel and click on Weelko logo to select the desired menus.
- The "MODE" menu allows you to choose between four programs. It is recommended to start with C and to select the other programs depending on patient's needs.

adjust its intensity by using the buttons "+" and "-".

- The "LEVEL" menu allows to adjust 3 inflation intensity levels.
- The "TIME" menu allows to adjust the treatment time. The default duration is 30 minutes.
- Press "PLAY/PAUSE" to begin or pause the treatment.

MAINTENANCE


- Connect and disconnect the tubes carefully to extend the product lifetime. However, this device is supplied with a blanket to store the cables so that they are all connected and disconnected at the same time. Only the general power cord at the rear part of the device should be connected and disconnected.
- After use, clean the suit with a wet cloth.

CAUTIONS

- Be careful not to wrap the airbag too tight, to avoid hurting the users when the air is too strong.
- Check that all the tubes are properly connected.
- The air pressure should not be set too high to avoid causing pain.
- When the machine is working, a professional must be next to it controlling it.
- Use a cellulose protector underneath the pressotherapy suit to ensure the best hygienic conditions.

ACCESSORIES

RIGHT LEG
LEFT LEG
RIGHT ARM
LEFT ARM
WAIST
RIGHT FOOT
LEFT FOOT
TUBES

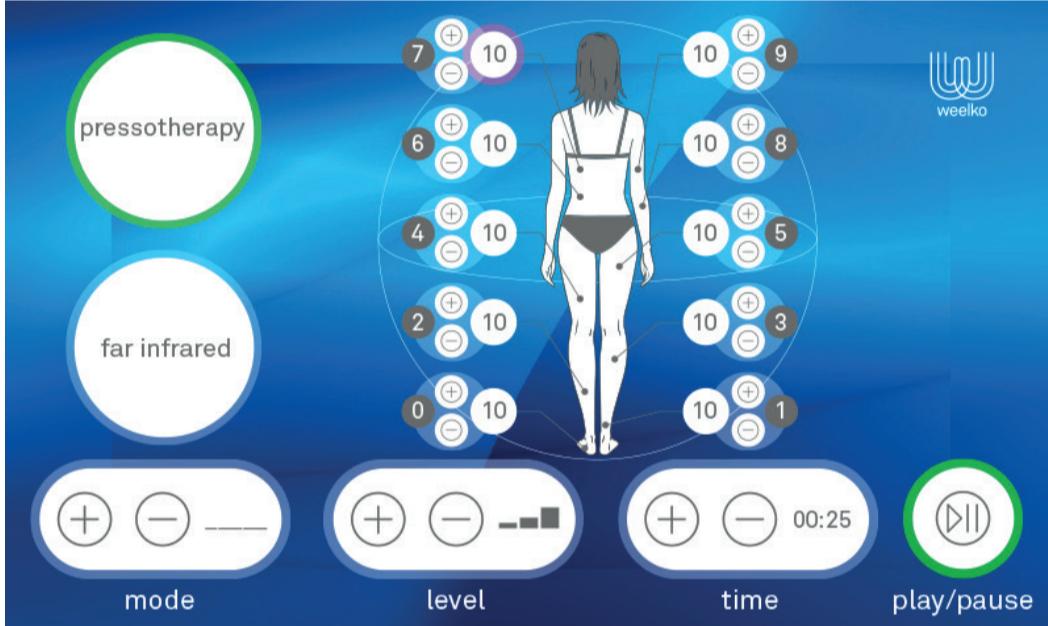TECHNICAL SHEET

Weights and sizes	Packing size (cm)	66x45x40
	Gross weight (kg)	18,5
	Product size (cm)	34x38x15
Working characteristics	Working power	50 W
	Air pressure	0-0,4 kg/cm ²
	Air flow	0,40 L/min.
	Rhythm of air pressure	0-15 s.
	Air channels	10 (20 bags)
Electrical characteristics	Rated voltage	220V ~ 240V
	Frequency	50Hz/60Hz
	Rated power	50 W

If you don't want to use a concrete mode, the device allows to choose sections separately and to adjust the inflation intensity of each section. You just need to press the area you want to treat and

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